

TABLE OF CONTENTS

App Installation	<u>3</u>
New User Registration	<u>4</u>
Set Up Your Dawn House Bed	<u>10</u>
<u>In-App Remote</u>	<u>15</u>
<u>Health Home Screen</u>	<u>17</u>
<u>Health Data</u>	<u>22</u>
Routines	<u>25</u>
Sharing: Dawn House Bed Users	<u>30</u>
Sharing: Family Accounts	<u>35</u>
Sign In & FaceID	<u>39</u>
Reset Your Password	<u>43</u>
Account & App Settings	<u>45</u>

APP INSTALLATION



ONIOS

In the iOS App Store, search for "Dawn House" or select the link below:

https://apps.apple.com/us/app/dawn-house/id1580189611



ON ANDROID

In the Google Play App Store, search for "Dawn House" or select the link below: https://androidapk-s.com/app/1580189611/dawn-house

NEW USER REGISTRATION


THERE ARE FOUR WAYS TO REGISTER:


- 1. Register by email
- 2. Register with Facebook
- 3. Register with Gmail
- 4. Register with Apple (iOS Users only)


After you will proceed to set up your Dawn
House bed or set up a family sharing
report-only account.

REGISTER BY EMAIL

- 1. To register with your email address, select the button "Sign up with email."
- 2. On the next screen, enter your email and create a password. Your password must be at least 8 characters and include any special character (ex. !@#\$%).
- 3. You must enter the same password twice then select the button for "Create my account."

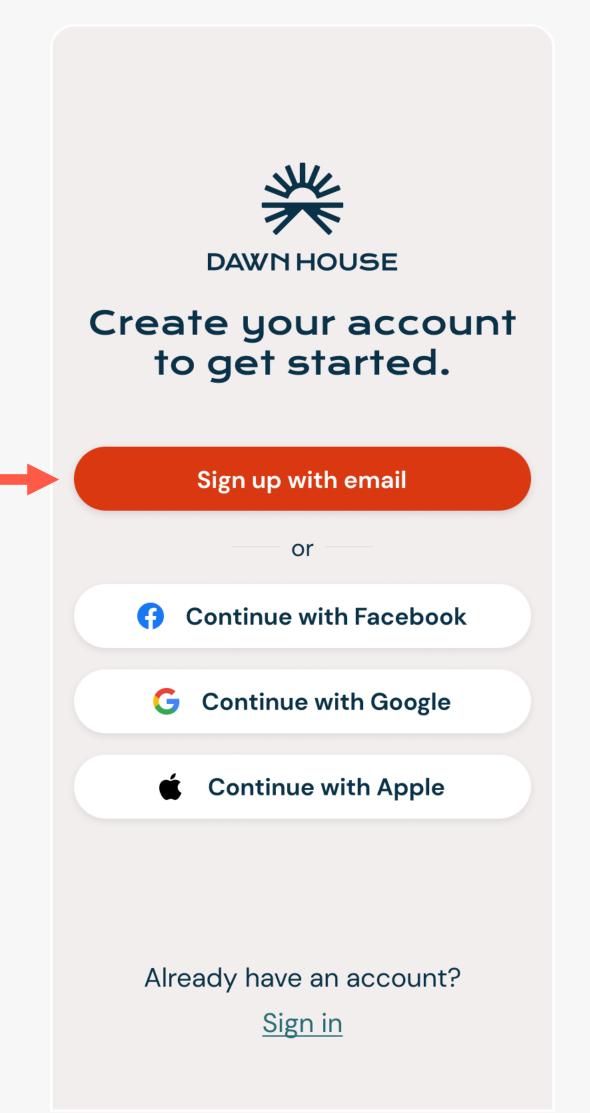


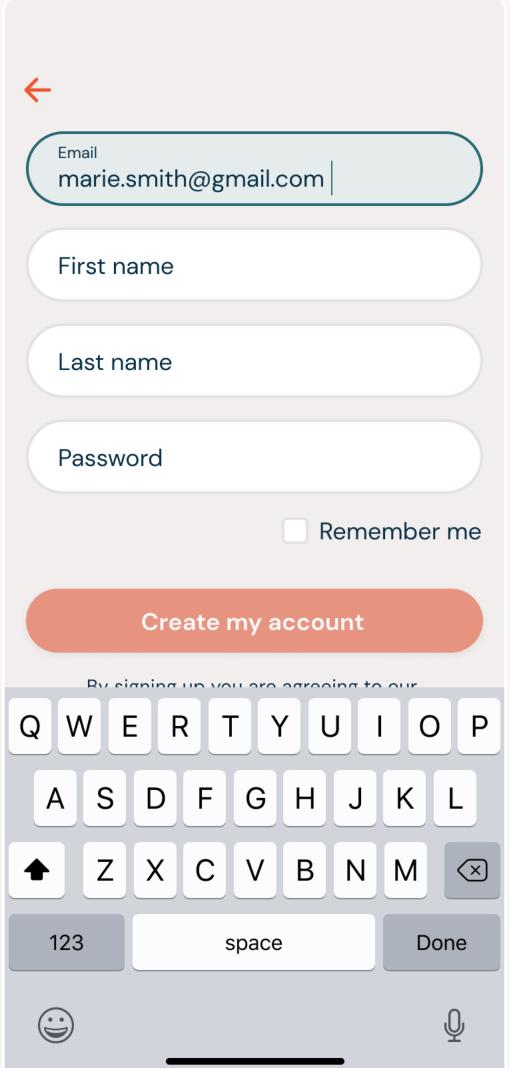


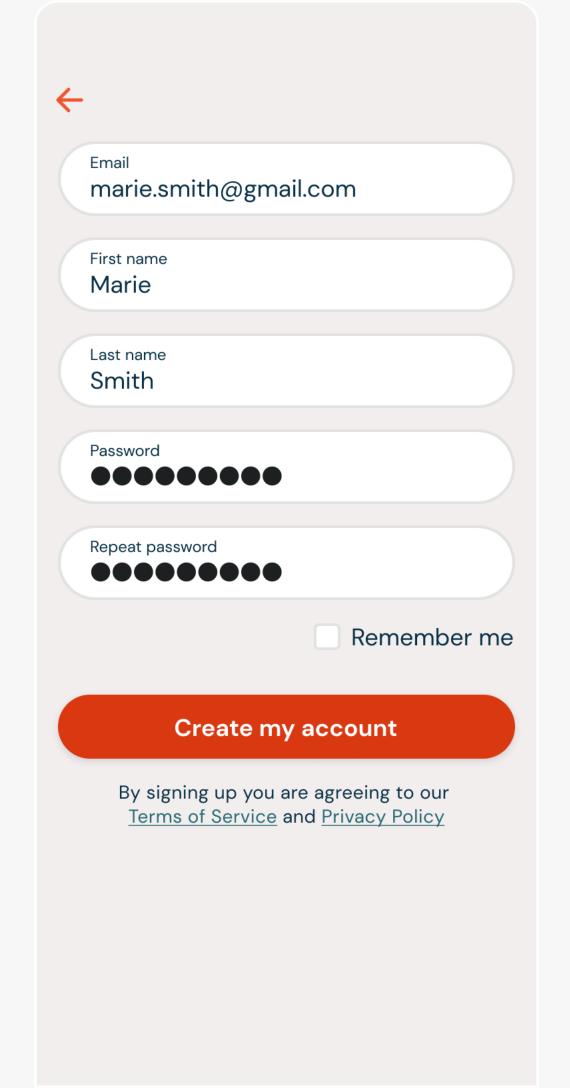

REGISTER WITH FACEBOOK


- 1. To register with your Facebook account, select "Continue with Facebook".
- 2. You will see the pop up modal asking permission to open Facebook. Select "Continue".
- 3. You will be prompted to log into your Facebook account and grant permission to Dawn House.

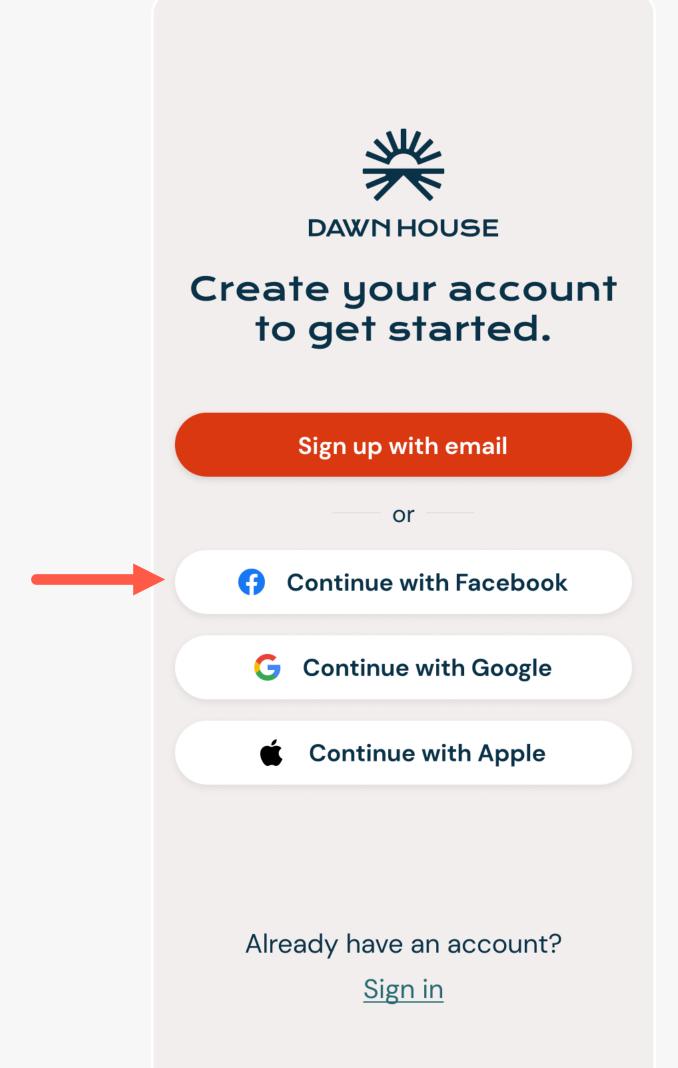


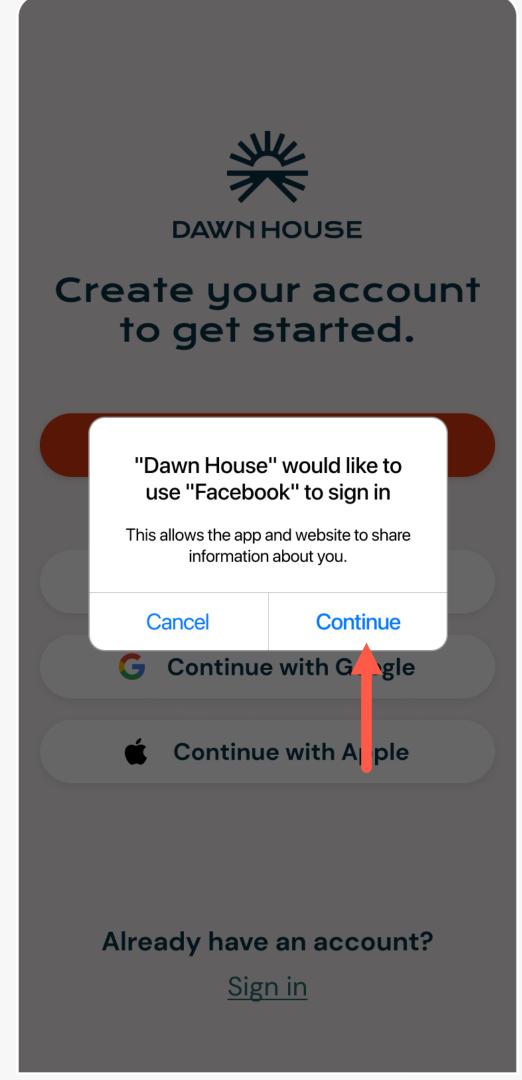

REGISTER WITH GOOGLE

- 1. To register with your Facebook account, select "Continue with Google".
- 2. You will see the pop up modal asking permission to open Gmail. Select "Continue".
- 3. You will be prompted to log into your Gmail account and grant permission to Dawn House.

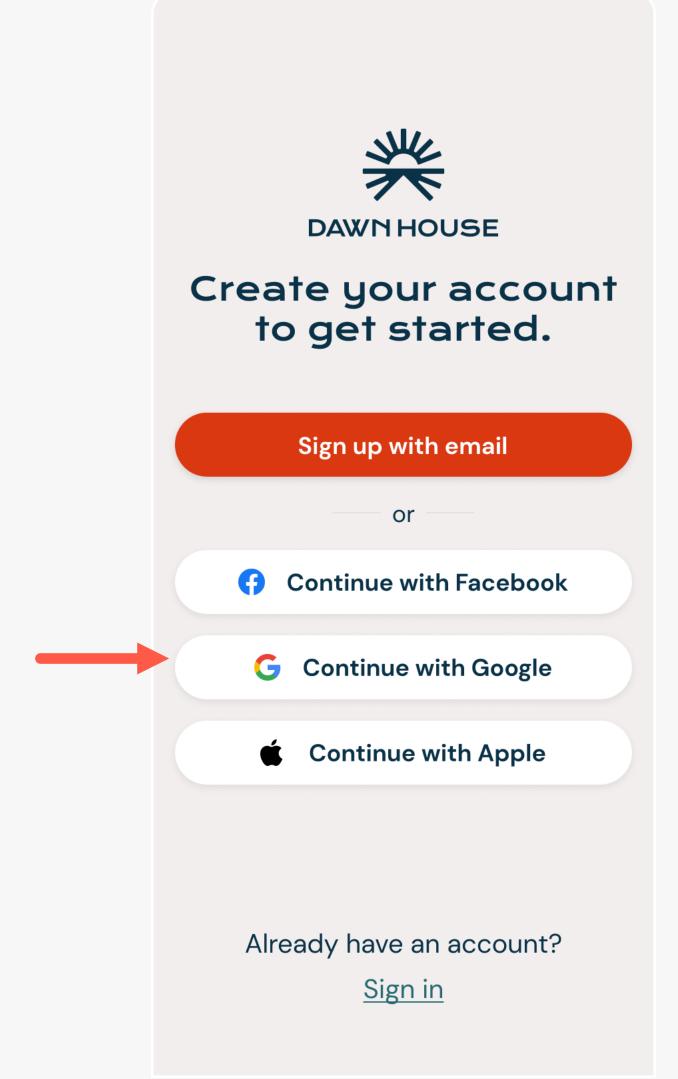


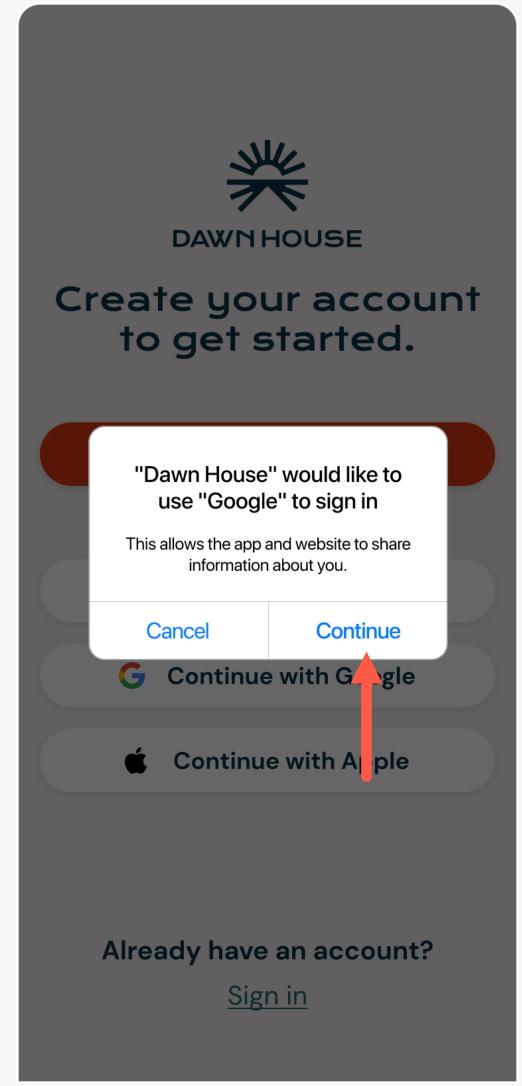


REGISTER WITH

APPLE (iOS Users Only)

- 1. To register with your Apple ID, select "Continue with Apple".
- 2. You will see the Apple menu expansion. You can choose to register with your Apple ID or with a hidden email that will forward to your Apple ID email.
- 3. Make your selection then press "Continue".

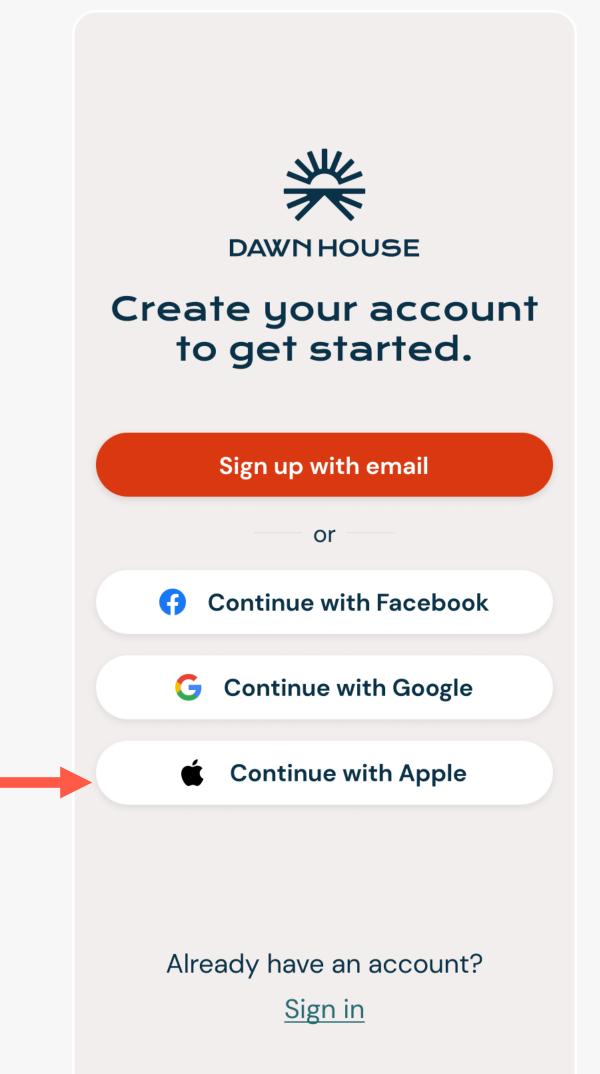


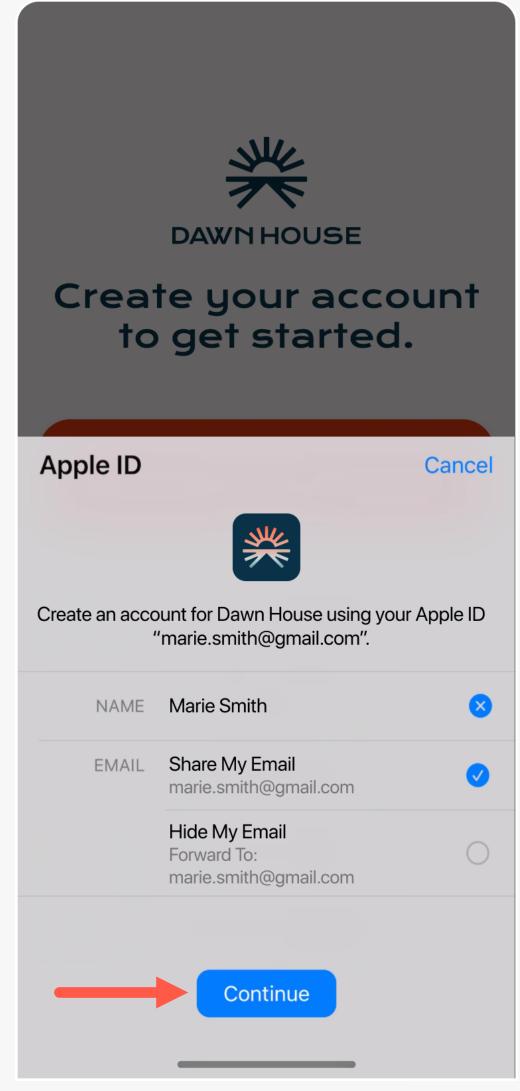

SELECT APP USE INTENT

If you have received your Dawn House bed or want to reconnect your existing bed, select the button for "Start bed setup". Proceed to Page 10 of the manual.

If you received an invitation to set up a Dawn House account to view a loved one's sleep report, select the button for "Create report-only account." Proceed to Page 35 of the manual.

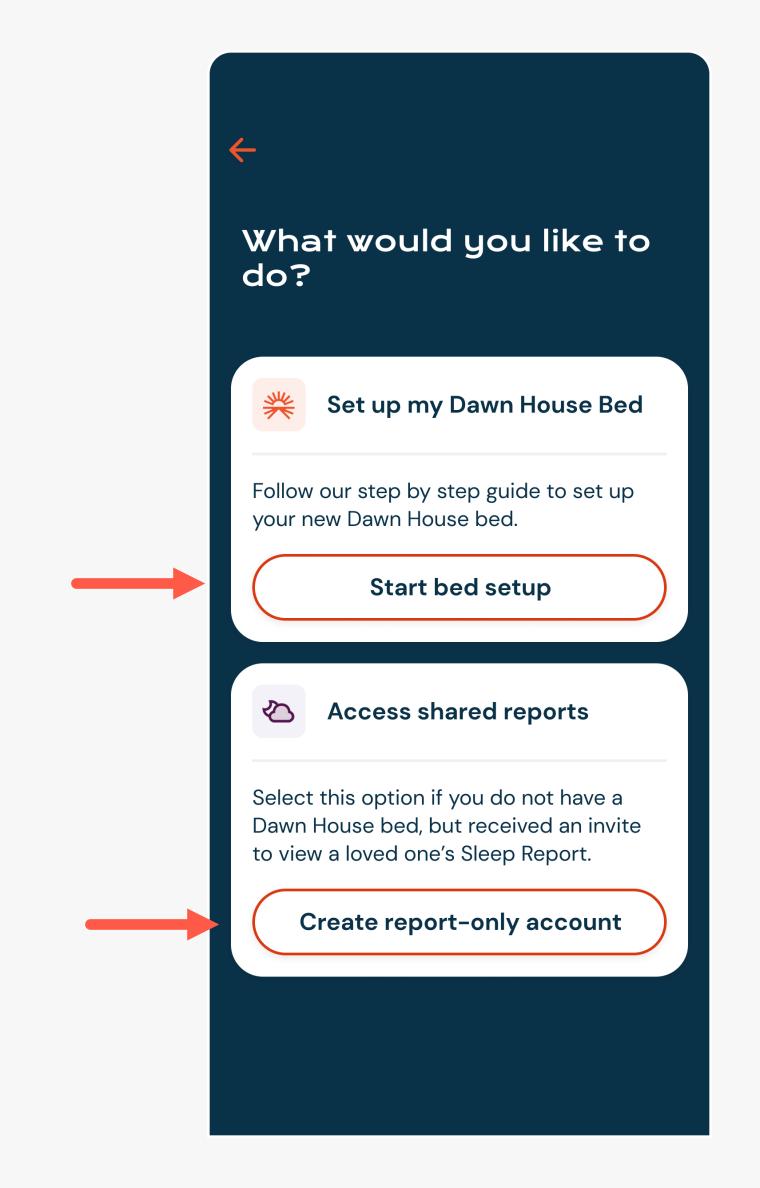


SET UP YOUR DAWN HOUSE BED


TO SET UP YOUR DAWN HOUSE BED:


- l. Answer the profile questions
- 2. Scan the QR code on your bed
- 3. Connect to your WiFi

ANSWER PROFILE QUESTIONS

After you register your account, you will be asked a few questions to calibrate your bed.

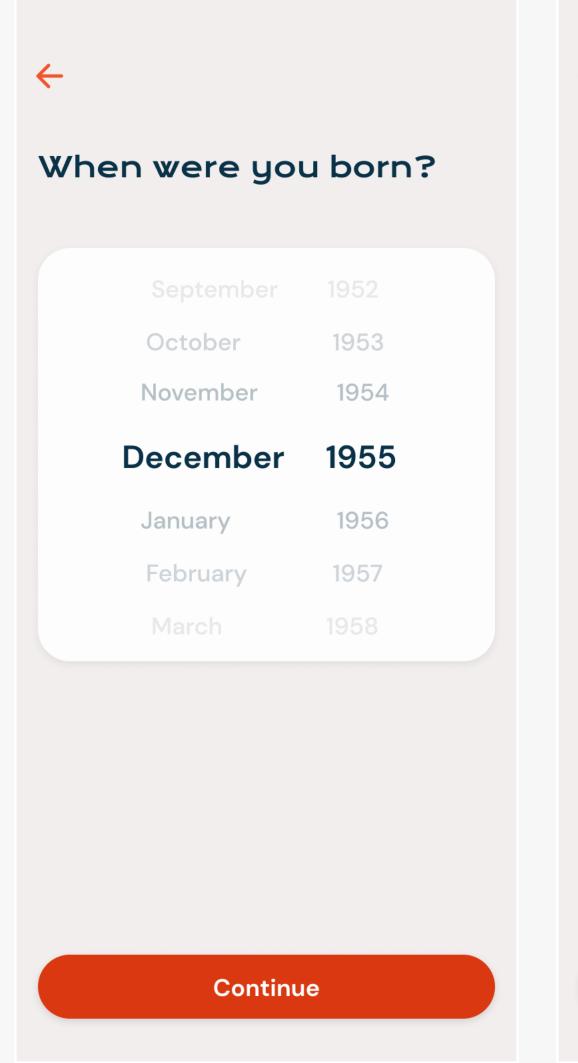

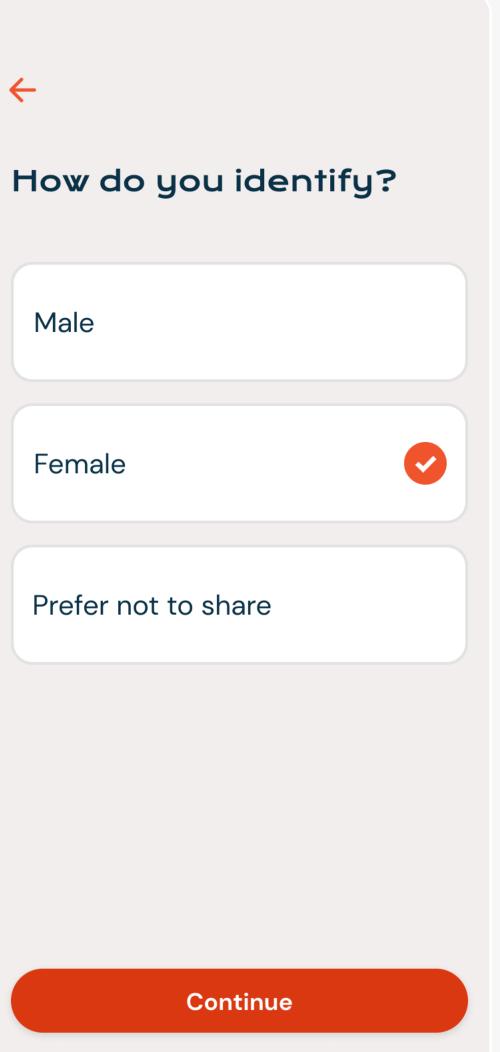

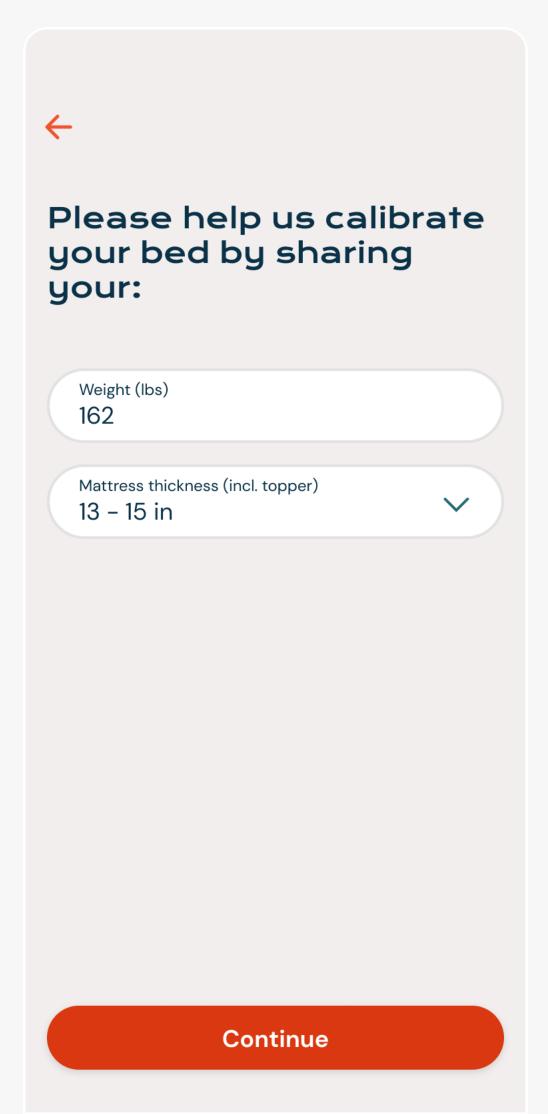

FIND YOUR QR CODE

You will need to find the QR code to scan and connect to the Dawn House app.

- 1. Shift your bed to a head raised position
- 2. Look behind the headboard on the bed base to find the QR code
- 3. The QR code will look like the image below, with a large symbol and a 16-digit character code beneath it:









SCAN THE QR CODE


Next you will be prompted to connect to your bed by scanning the QR code located on your bed base.

You will first see a prompt to allow access to your phone's Bluetooth and Location settings. This is needed to bind your connection to the bed.

After, scan with your camera or manually enter the QR code located on your Dawn House bed base.

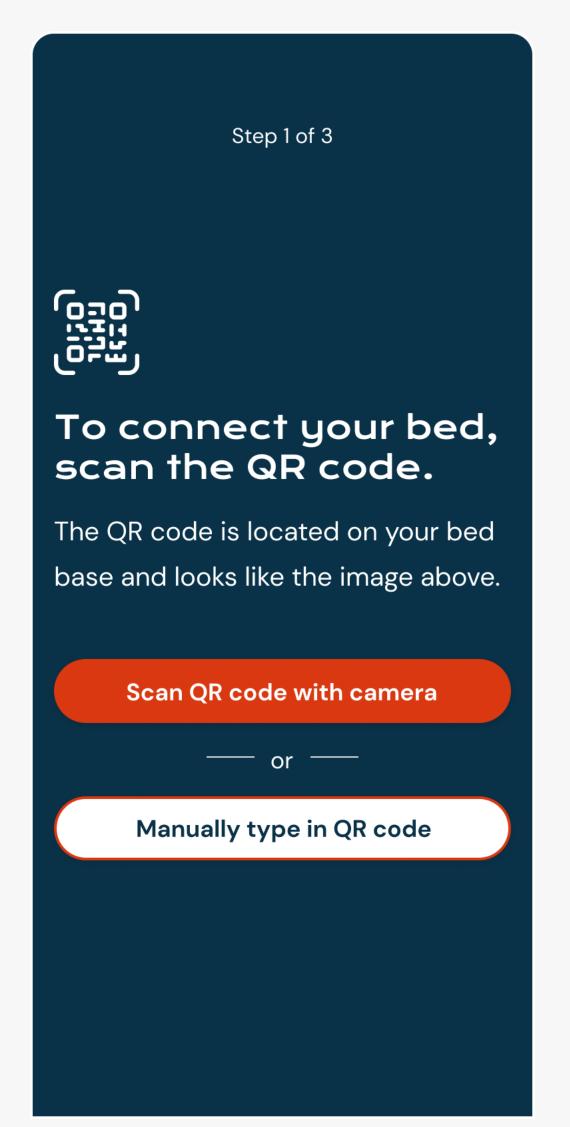

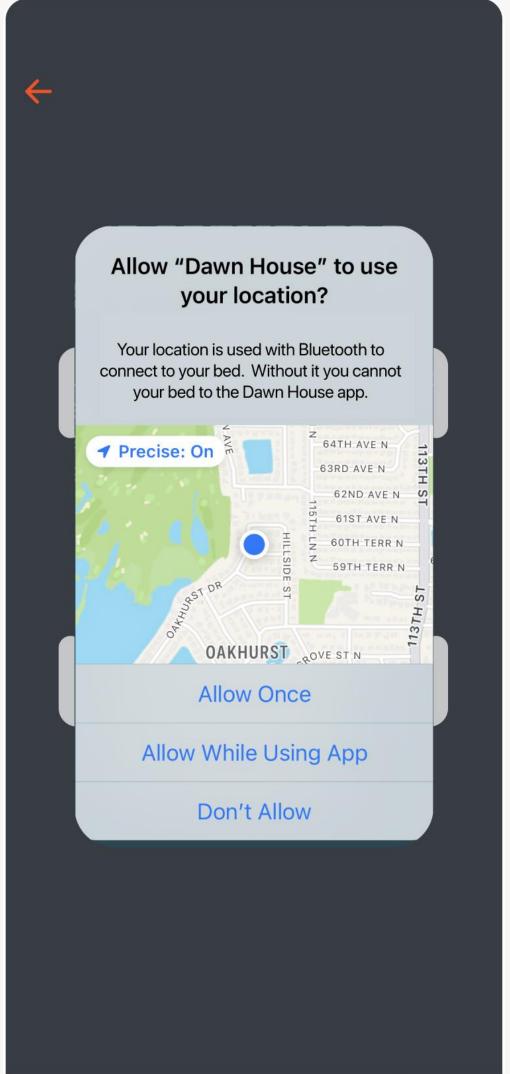

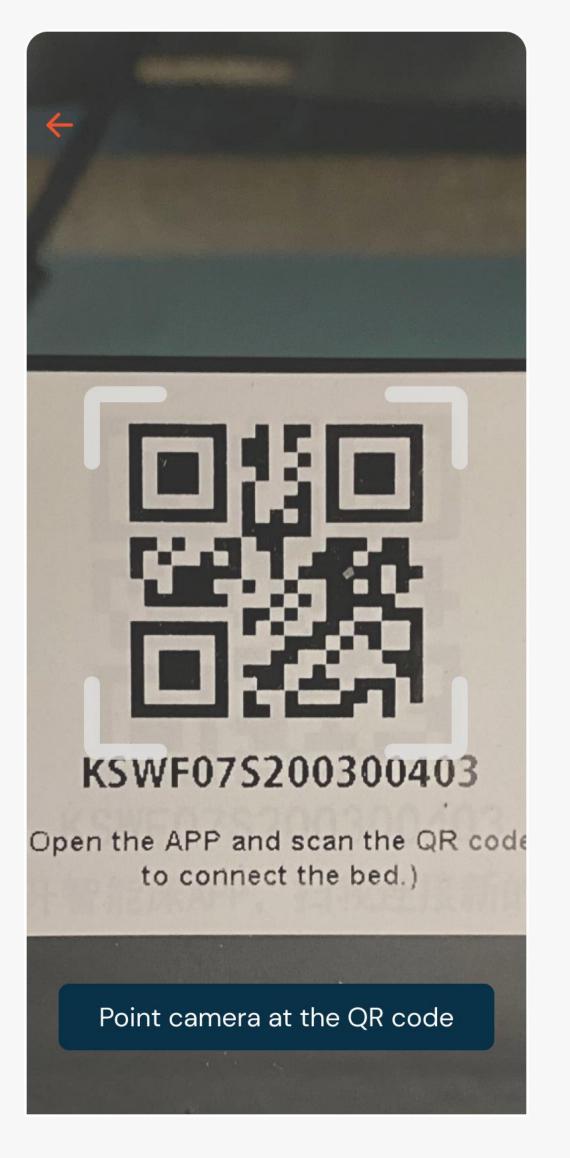



CONNECT TO YOUR WIFI

After you scan the QR code you must enter your Wi-Fi password to complete the connection to your Dawn House bed.

The app will ask for the Wi-Fi password that you are currently connected to.

Please make sure to connect to the Wi-Fi network with the strongest signal in your bedroom. To change the network, tap the "Select another network" link to choose another.

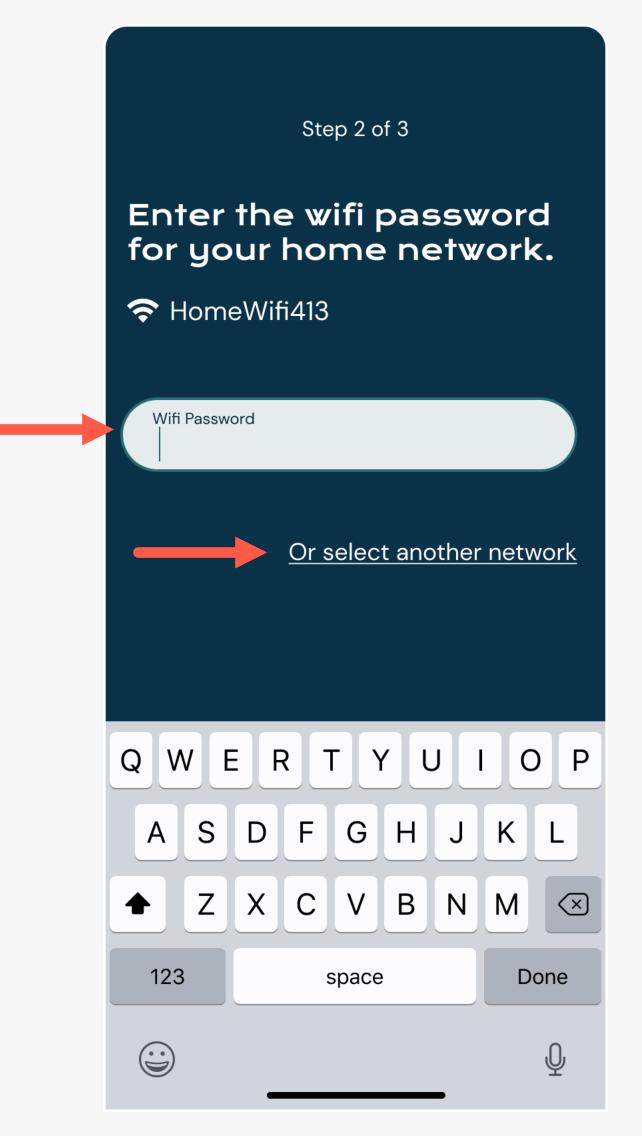

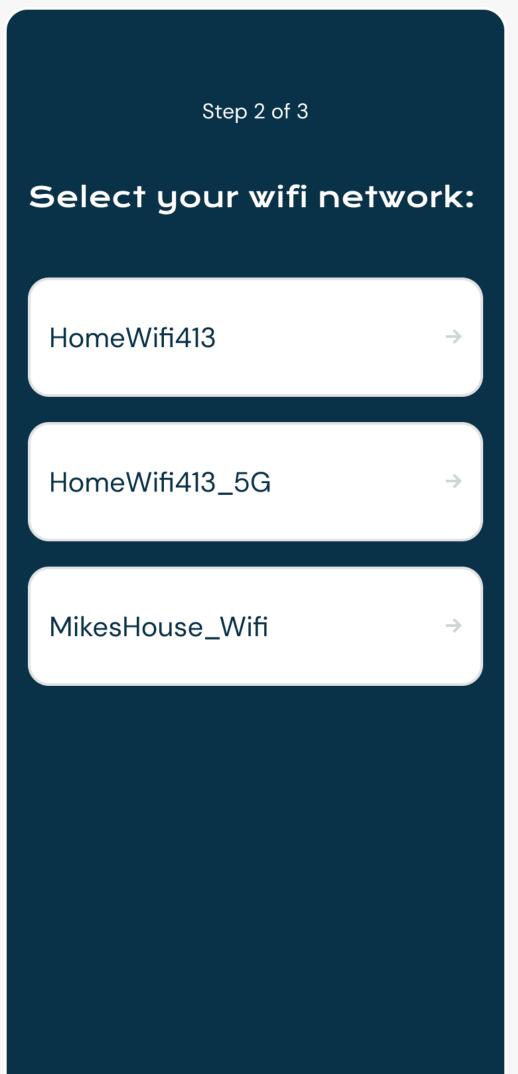

IN—APP REMOTE

You can operate your Dawn House bed with the physical remote or the in-app remote.

Please note: You must use your physical remote to set bed presets.

ACCESS YOUR

REMOTE


Tap the blue circle button in the bottom navigation to open the in-app remote. This will enable you to control your bed from your phone.


Here you can use the in-app remote to operate your bed. To close the remote, select the "Close" button in the top right.

For more help, you can open the in-app remote tutorial from the link "View remote tutorial \rightarrow "

Or if you prefer, you can reference the One Page Remote Cheat Sheet that came with your bed, or access the full Remote User Guide below:

https://dawnhouseliving.com/pages/manualsand-instructions

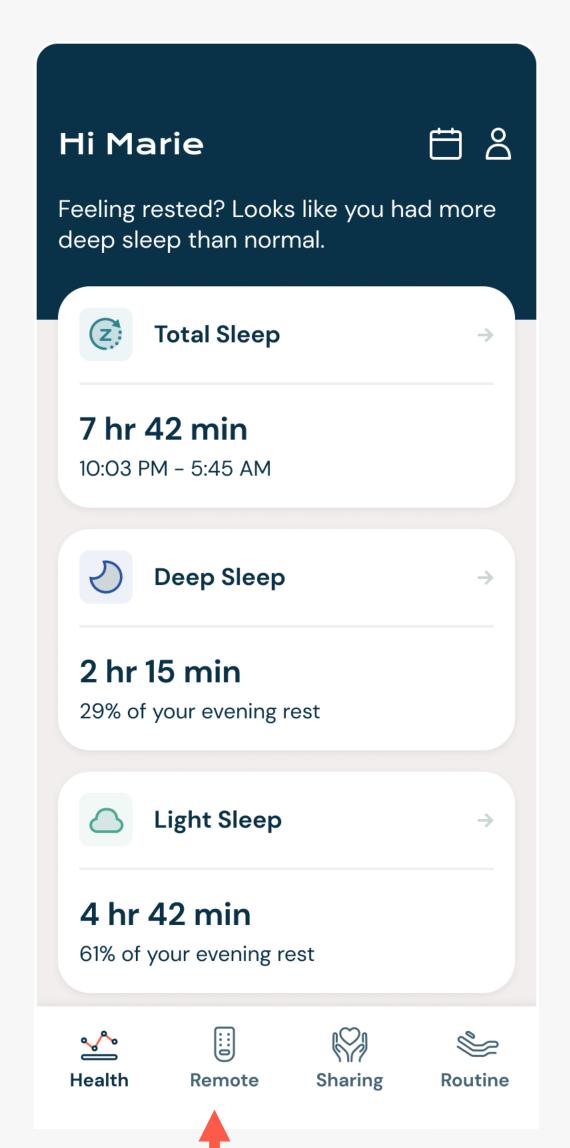

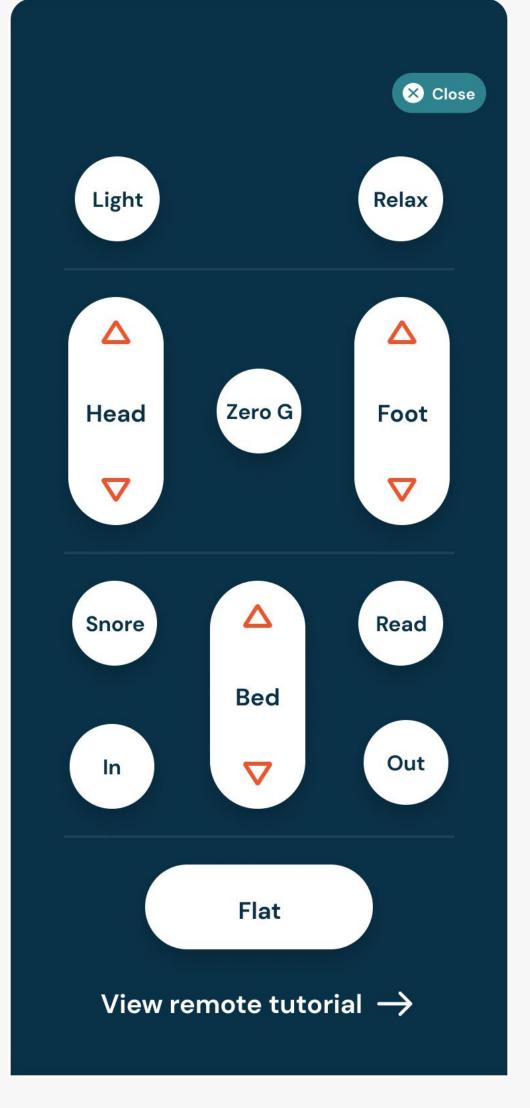


HEALTH HOME SCREEN


Once you into log into the Dawn House app, you will be automatically redirected to the Health Home Screen. Your view here will change with each night of rest.

YOUR HEALTH HOME SCREEN: FIRST VISIT

When you first set up the Dawn House app you can lay down and test out your bed and see the real-time data metrics for your heart and respiratory rate.

Your health dashboard will grow and change as additional data points are collected.

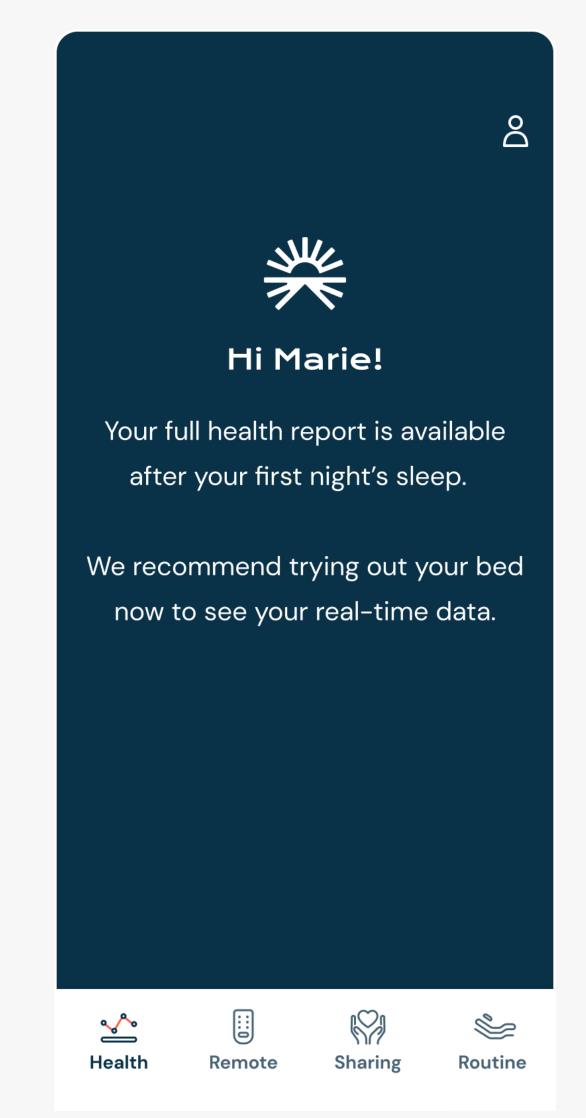

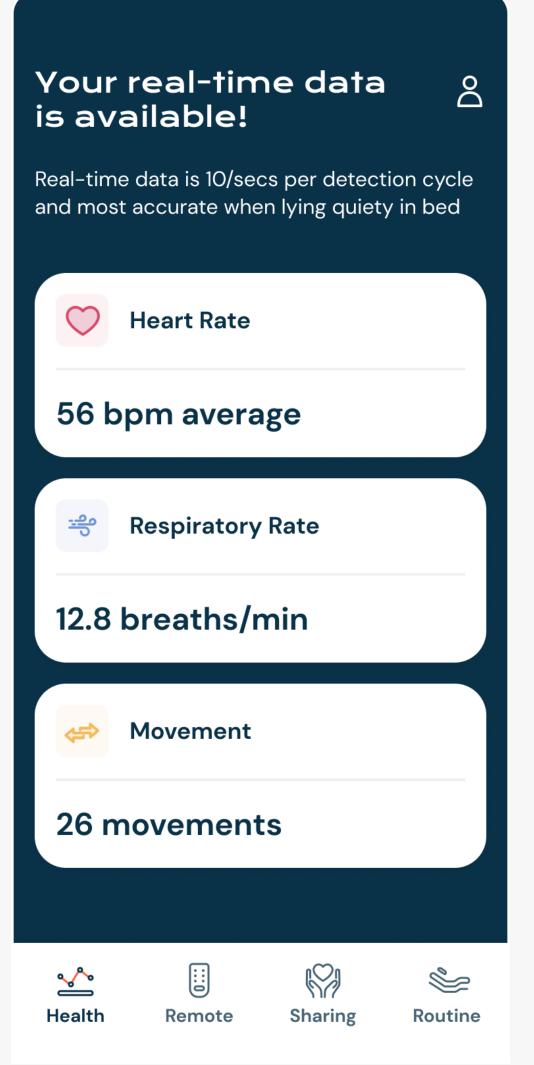



YOUR HEALTH HOME SCREEN: SLEEP REPORT

Your Dawn House bed includes state-of-the-art in-home wellness monitoring that measures key aspects of your sleep cycle and other important health metrics.

After your first evening's rest, your sleep report will be generated. With time, you will be able to identify patterns and help manage your health.

Click on any health card to see more details about this specific health metric.

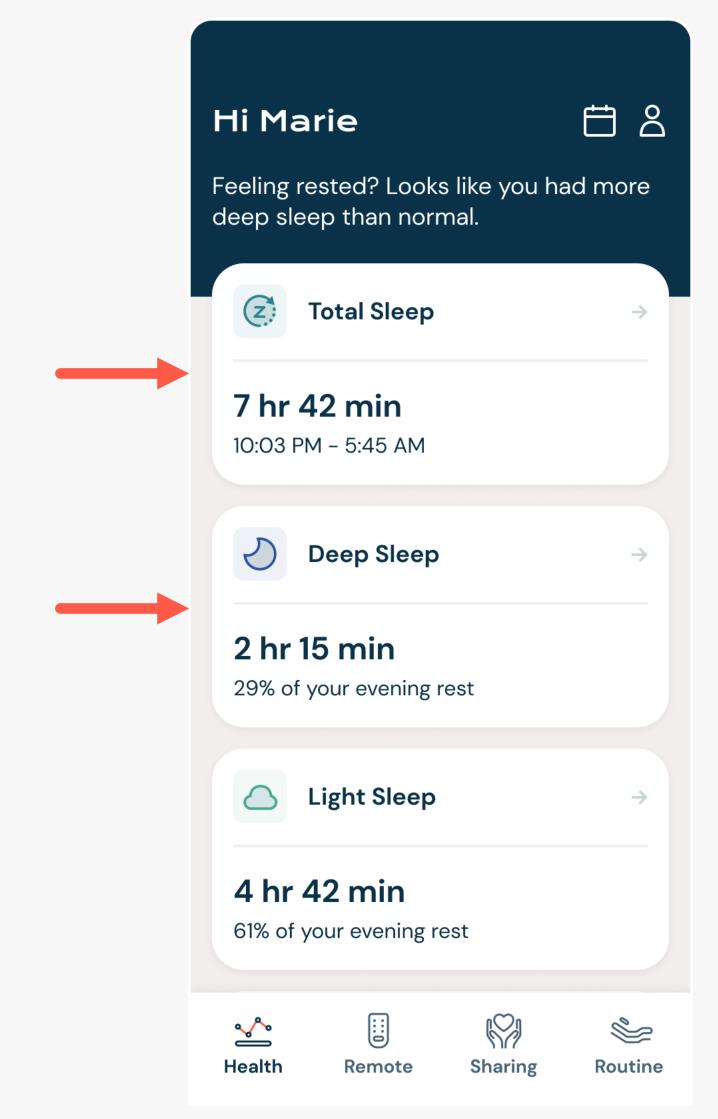

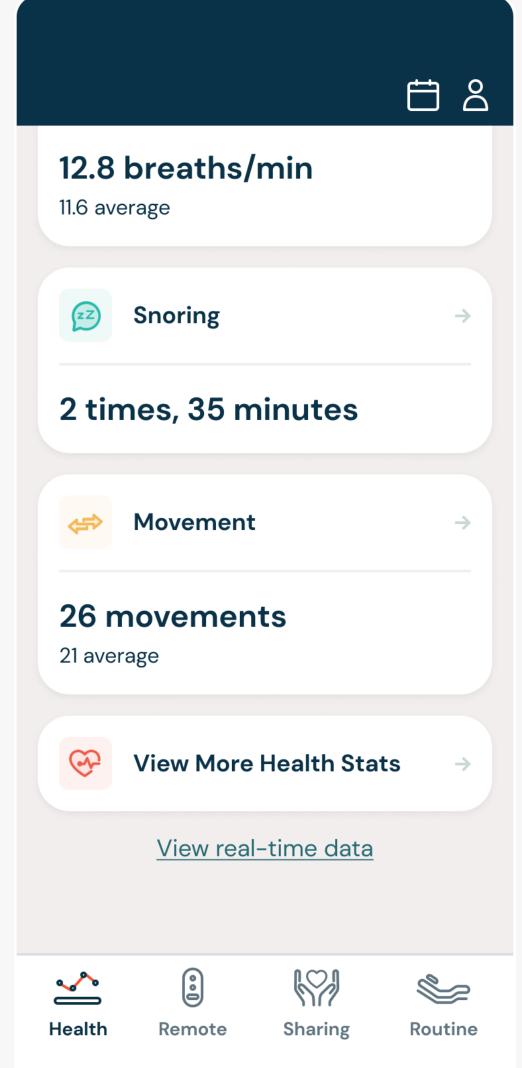



YOUR HEALTH HOME SCREEN: REAL-TIME DATA


Scroll to the bottom of your sleep report and select the link for "View Real-Time Data."

If you are seated on your bed, you will see the real-time metrics for your heart and respiratory rate.

If your bed has only been authorized to you, you will only see your real-time metrics. If your bed has been authorized to your partner, you will see real-time data for both of you.

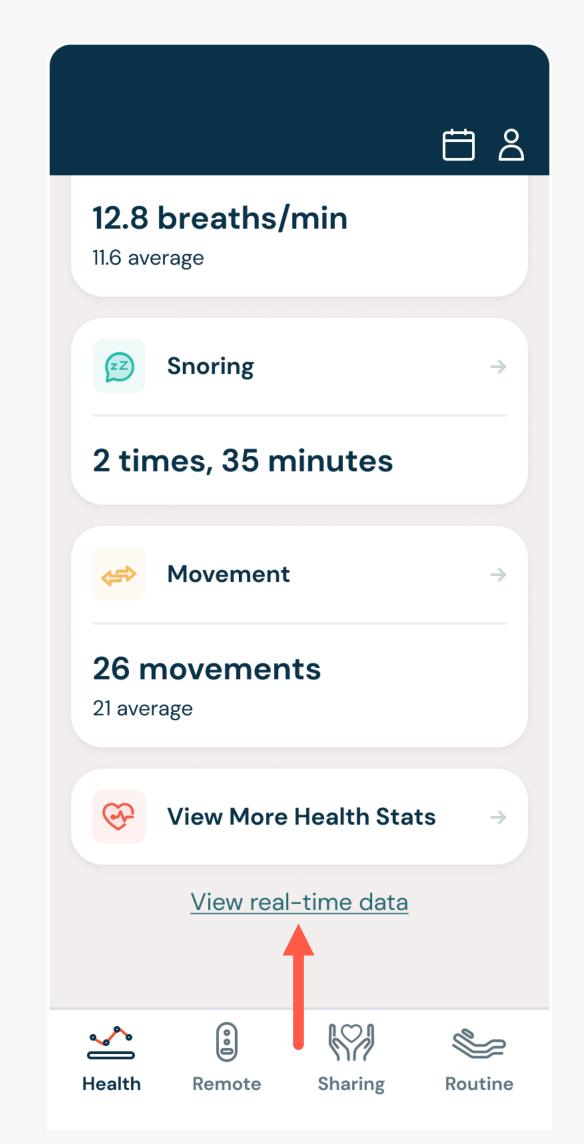

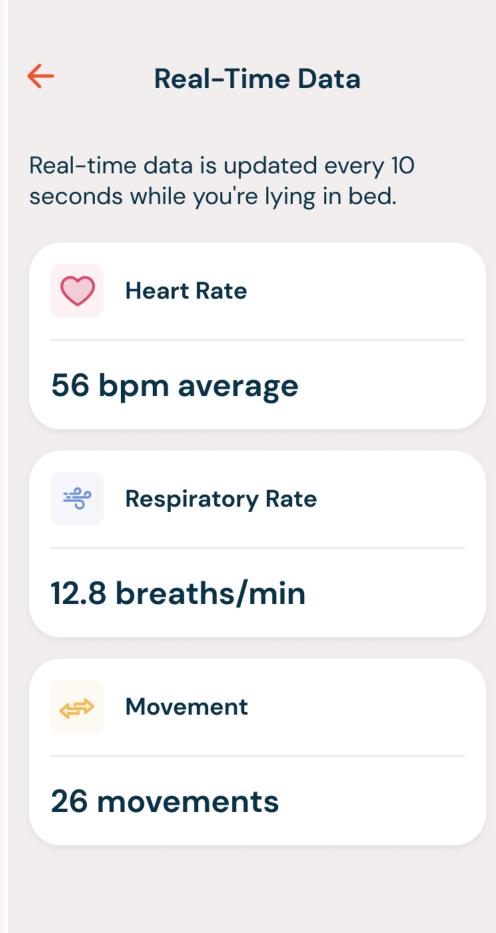


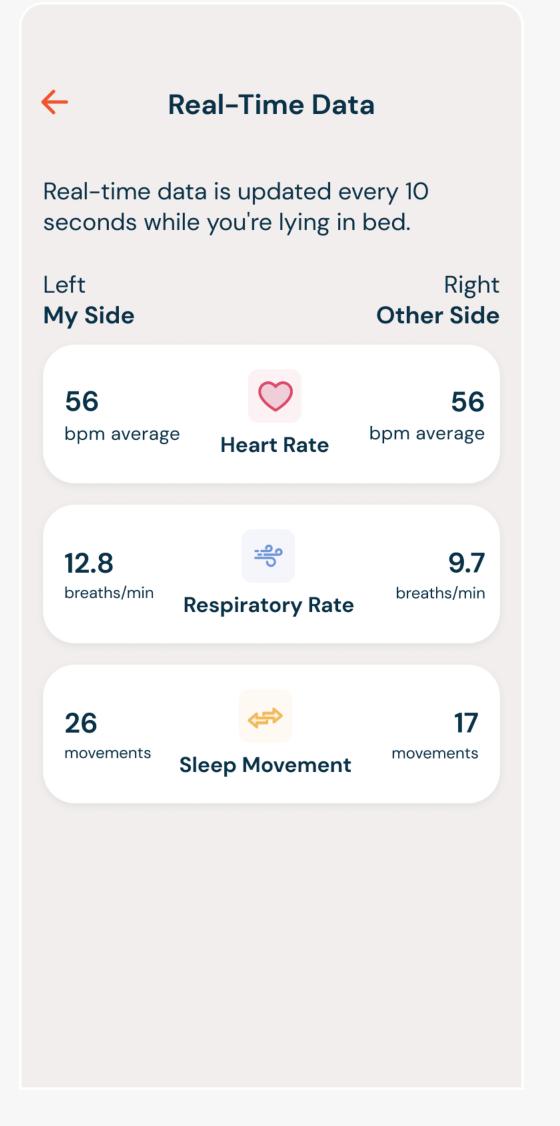


CALENDAR: VIEW
PAST HEALTH
METRICS

To view historical Sleep Reports, click the Calendar icon in the top right of the main Health Home screen.

This will open the calendar view. Use the <> arrows to navigate through the dates to select another date you would like to view data for. Then select "View Health Data."

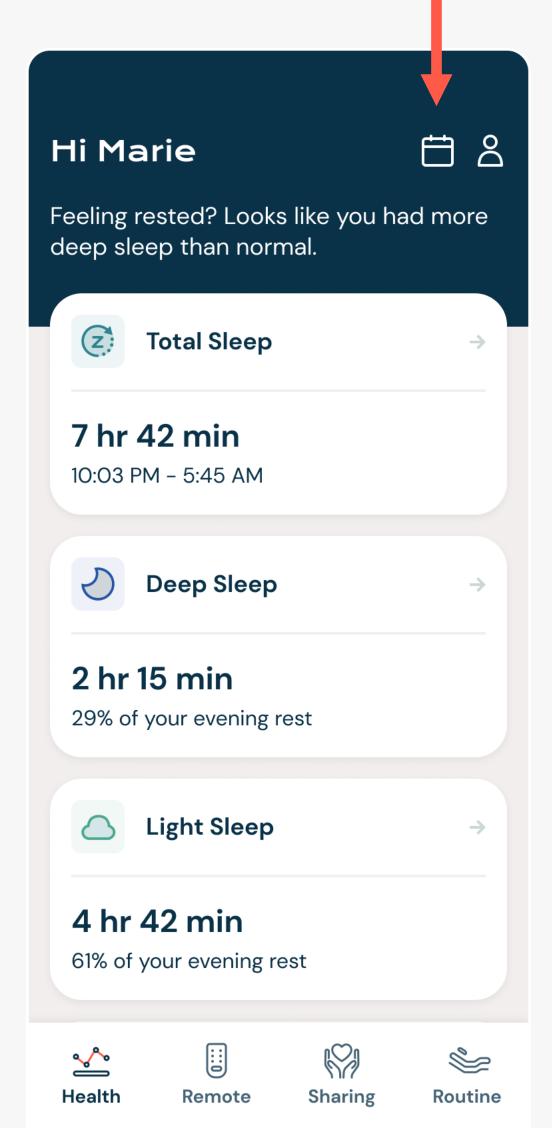

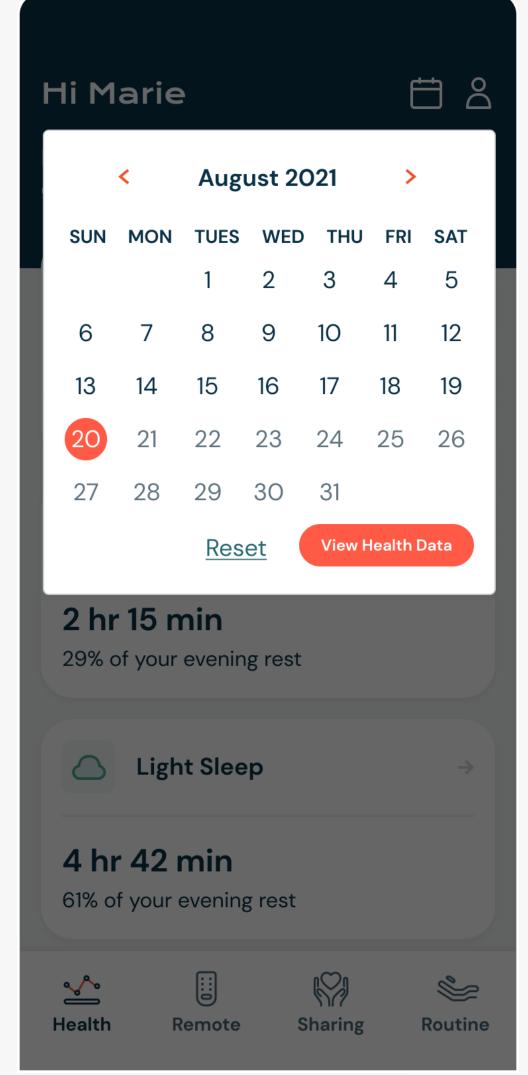

HEALTH DATA

The Dawn House app gives you access to vital health indicators generated while you rest. Each morning you will be able to access a Sleep Report sharing information about your sleep quality and duration, heart rate, respiratory rate, and more.


To access, tap on any of the white cards from the main Health Home Screen.


HEALTH DATA POINTS

Your health data includes:

- Total, Deep & Light Sleep
- Heart Rate
- Respiratory Rate
- Snoring
- Movement
- More Health Stats

To navigate between your health data, swipe the screen left or right.

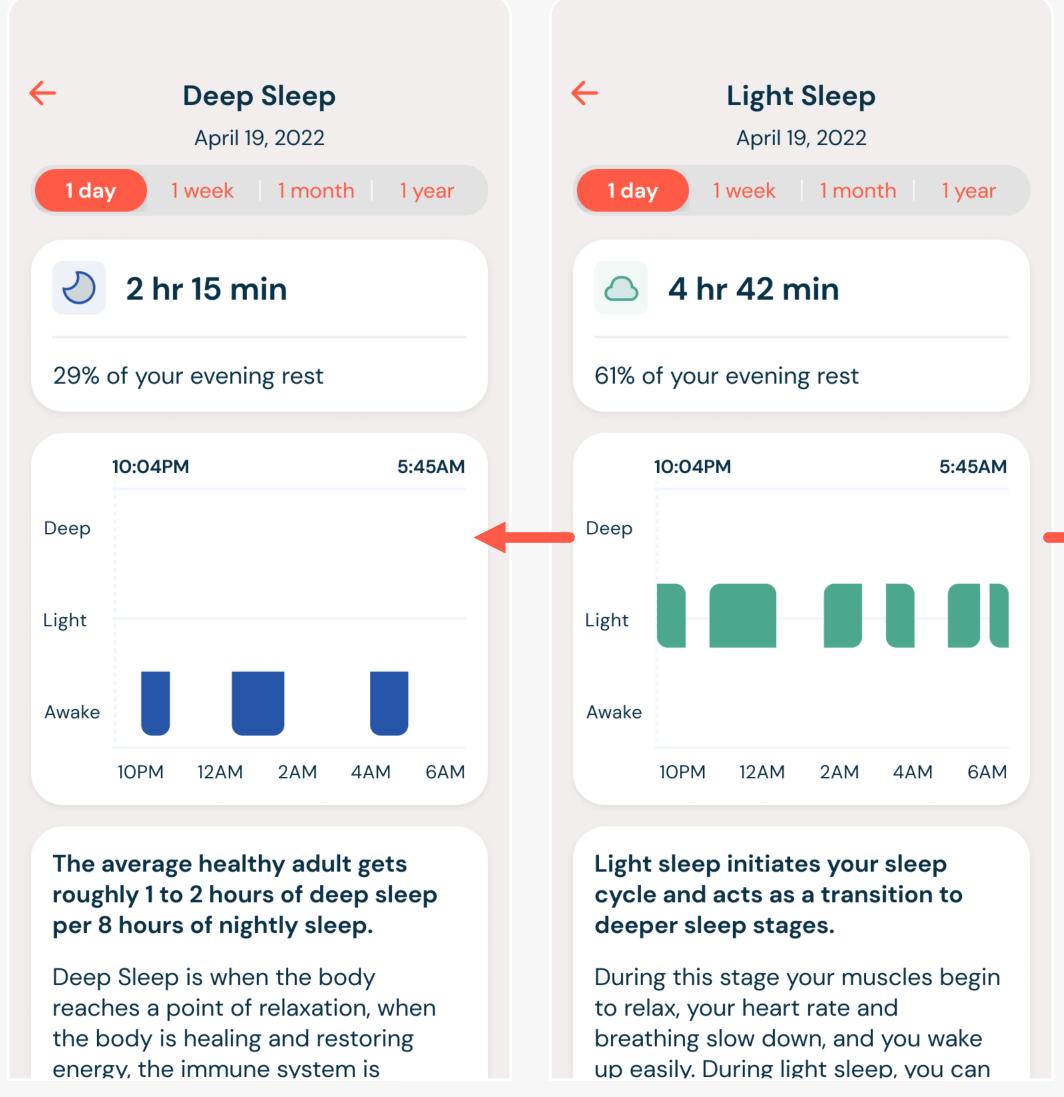

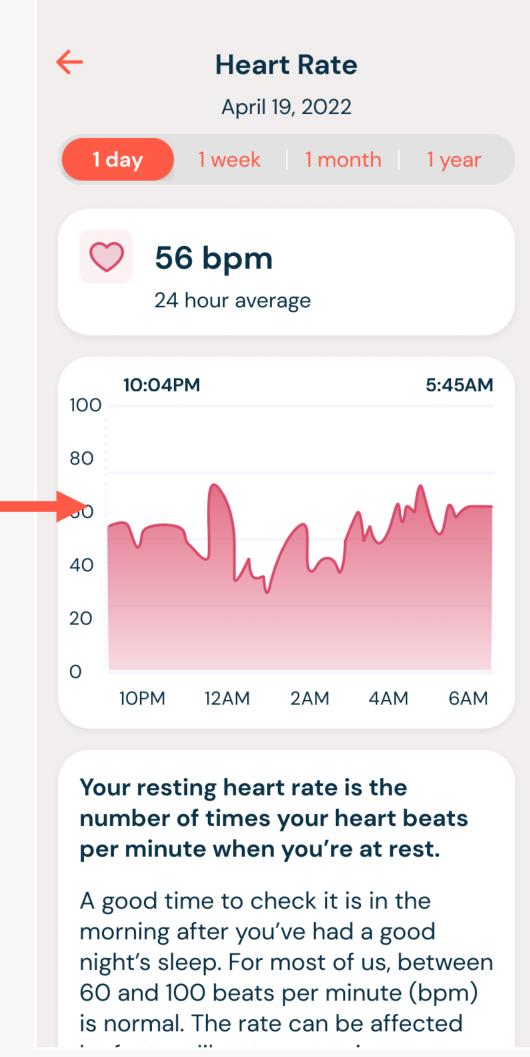


DAILY, WEEKLY & MONTHLY VIEWS


All of your important Sleep Report metrics are organized into weekly and monthly charts to make it easy to see trends and make adjustments as needed.


By default, you will see the Sleep Report metrics collected from the previous evening.

To view your weekly data trends, select "One Week."

To view your monthly trends, select "One Month."

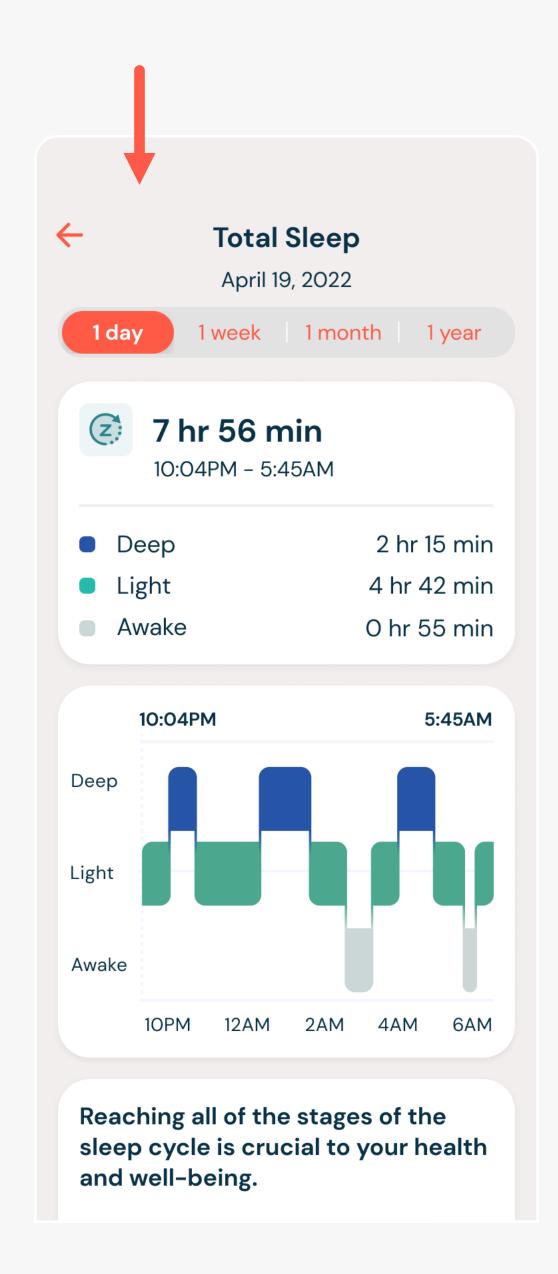

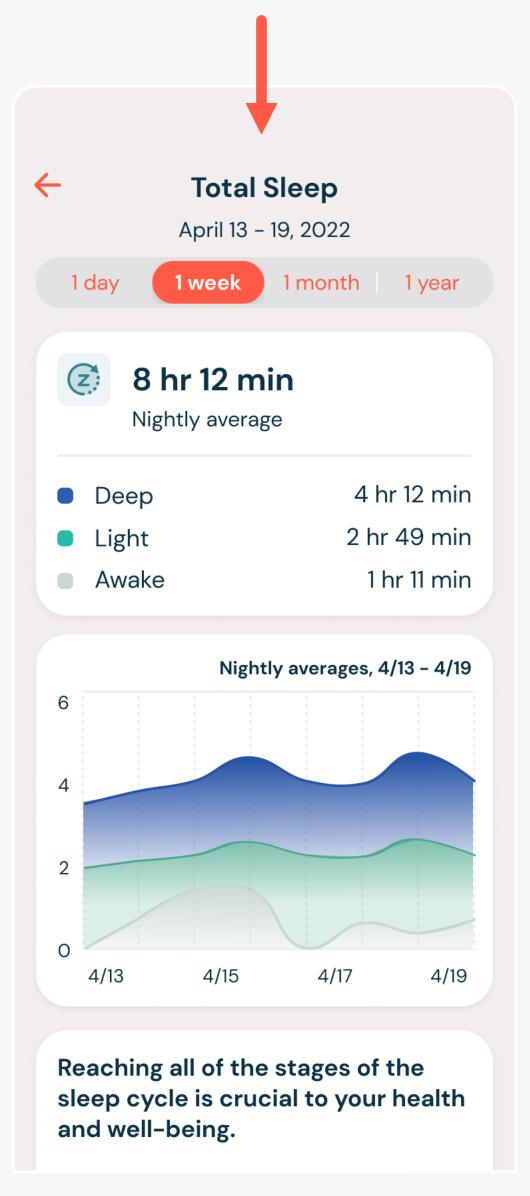

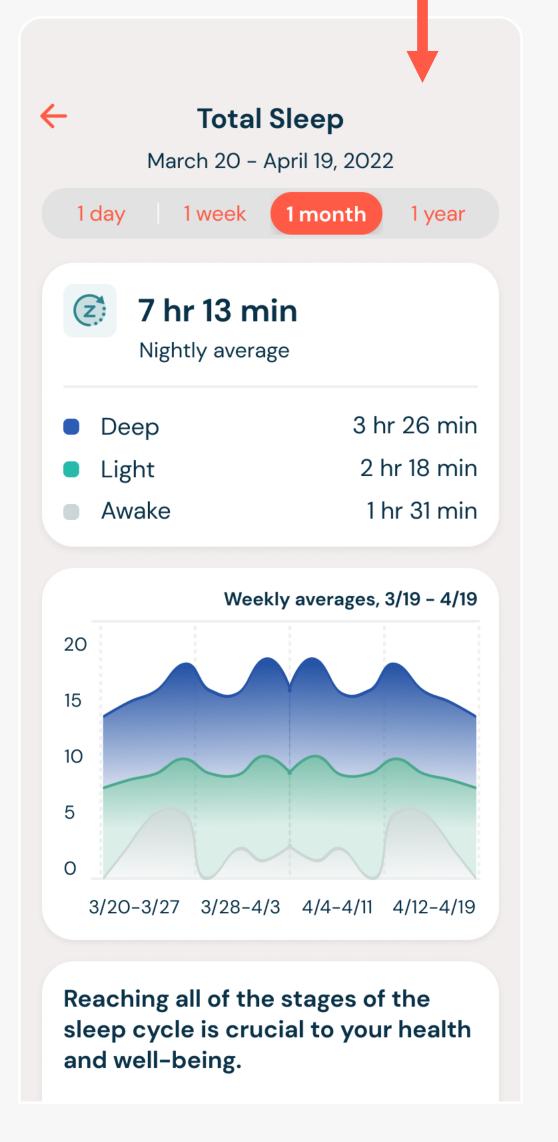

ROUTINES


It's easy to set alarm routines that activate specific functions of the bed and can be customized daily or set to repeat with any frequency.

YOUR ROUTINE HOME SCREEN

The Routine screen is accessed by selecting the clock icon in the bottom right of the navigation bar.

Here you can set up morning or evening routines for your Dawn House bed to automatically shift into the position of your choice at the time you schedule.


You can also turn on or off your underbed lighting at the scheduled time and auto correct your position when snoring is detected during sleep.

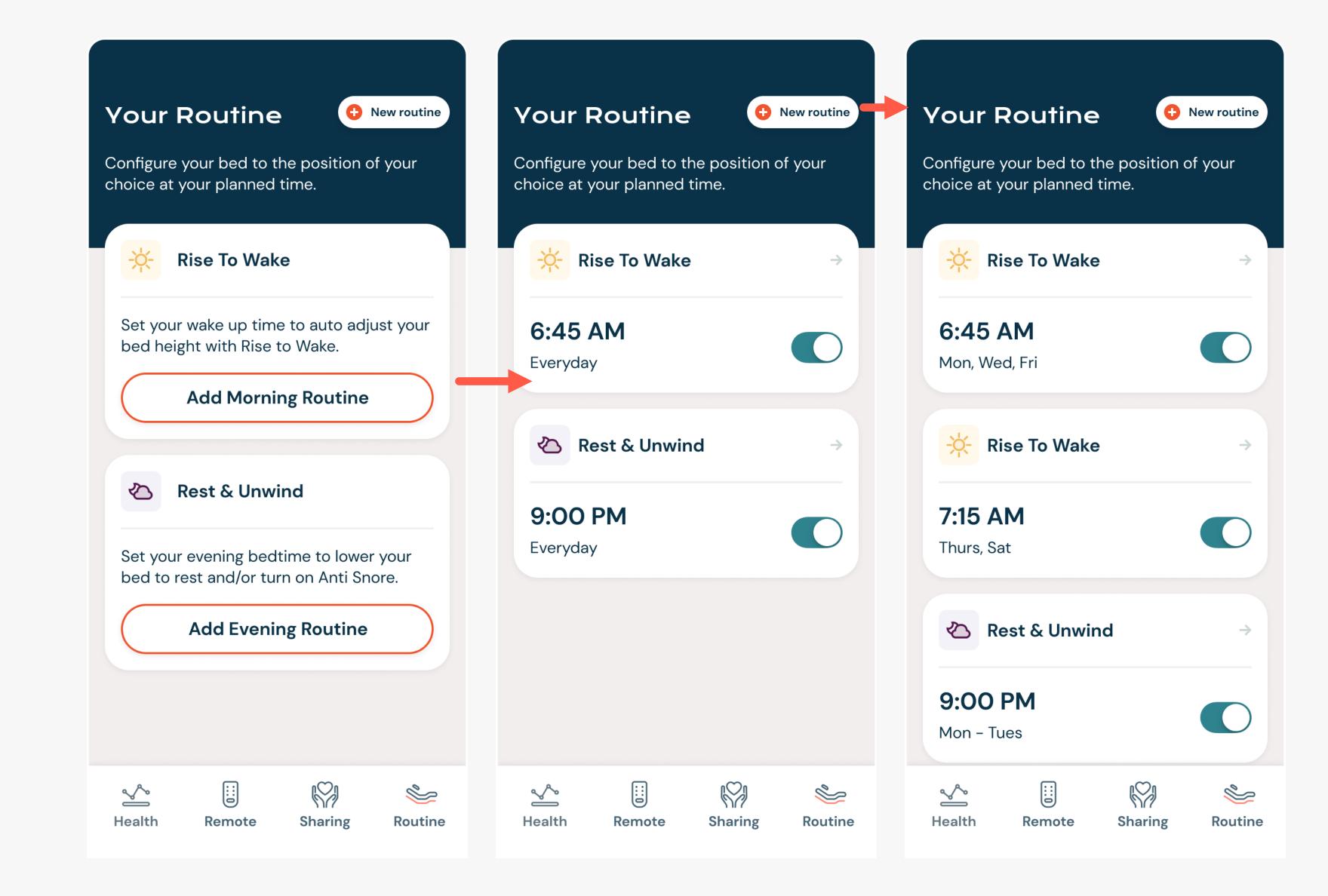



RISE TO WAKE

Whether you're up with the sun or like to sleep in, the Dawn House bed can gently elevate your head at an appointed hour, easing you into your new day with "Rise to Wake". To set your routine:

- 1. <u>Set Your Wake Up Time:</u> Use the timer wheel to select.
- 2. Set the Routine Frequency: Tap the "Repeat Routine" card to open and select how often you want to repeat this routine.
- 3. Turn Underbed Lighting On/Off: If you wish you can turn your underbed lighting on or off at the time of your routine. Select "No change" if you do not want to adjust the underbed lighting.

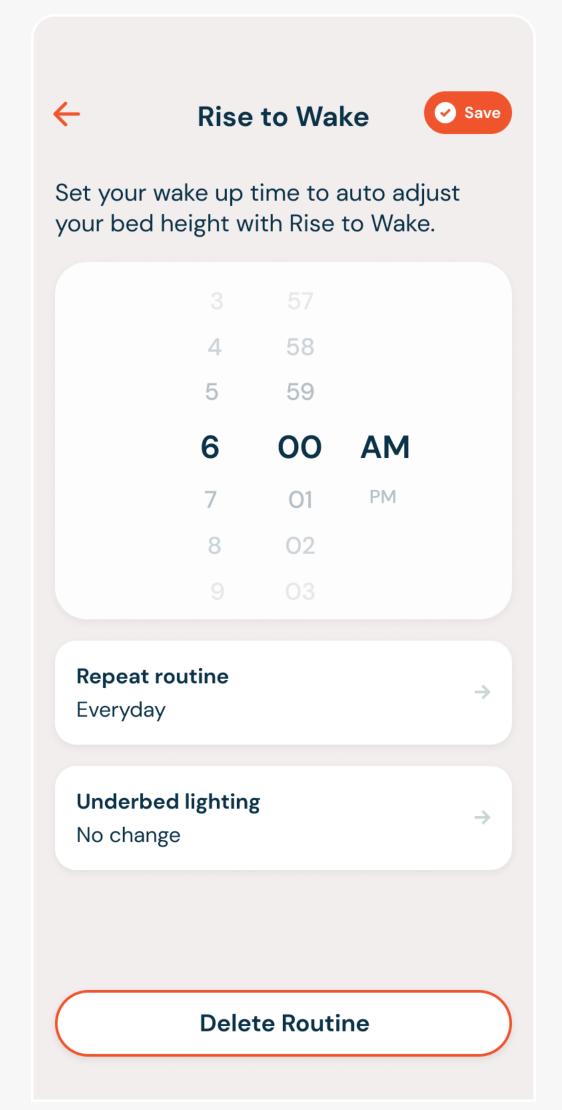

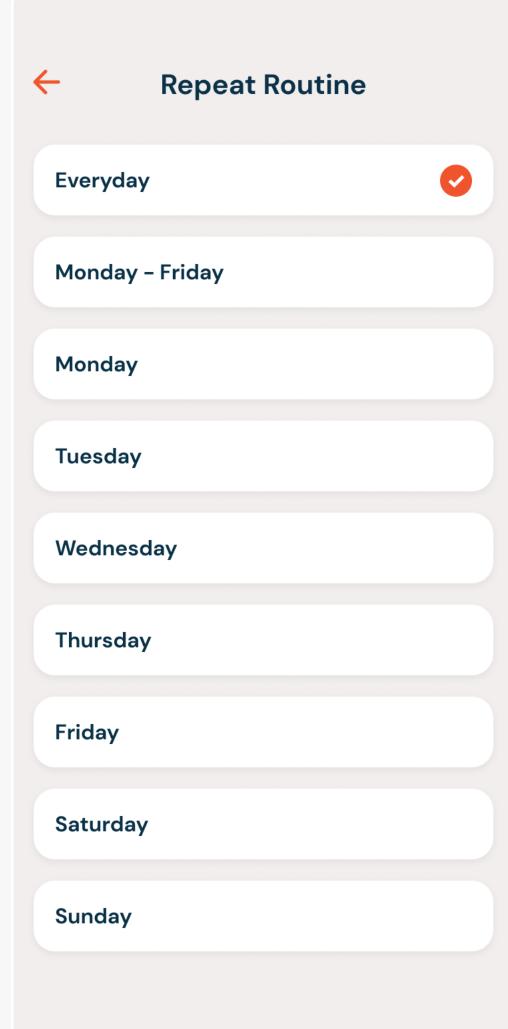


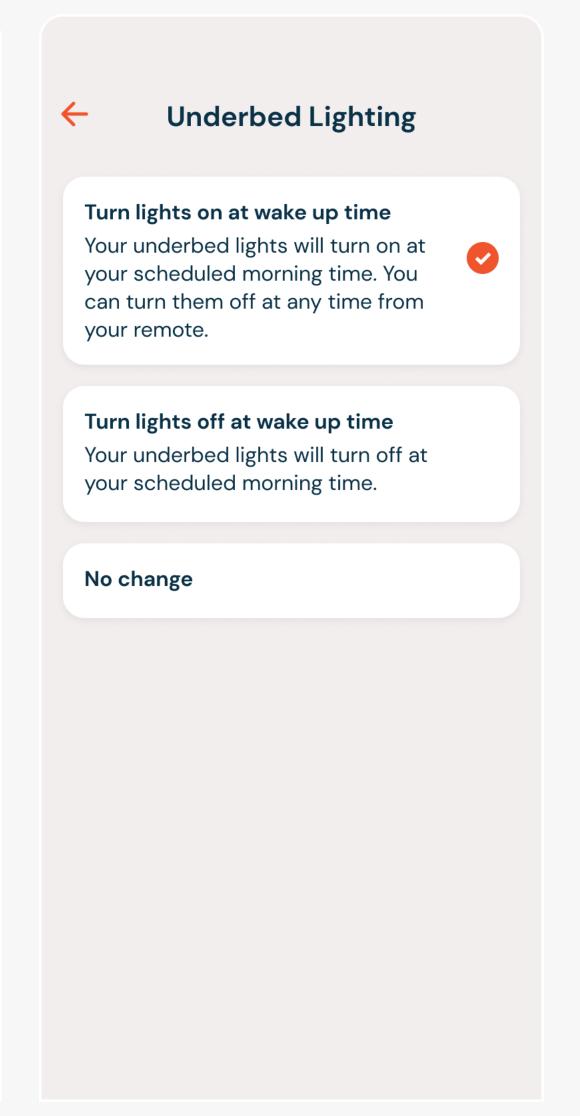


REST & UNWIND

Establish an evening routine with "Rest & Unwind." To set your routine:

- 1. <u>Set Your Bedtime:</u> Use the timer wheel to select.
- 2. <u>Set Your Bed Position:</u> Tap the "Position to Rest" card to set your bed position
- 3. Set the Routine Frequency: Tap the "Repeat Routine" card to open and select how often you want to repeat this routine.
- 4. Turn Underbed Lighting On/Off: Turn your underbed lighting on or off at the time of your routine. Select "No change" if you do not want to adjust the underbed lighting.
- 5. Enable Anti-Snore: Turn anti-snore on or off. When on, sophisticated sensors detect the onset of snoring and gently raise the head to promote more restful sleep.

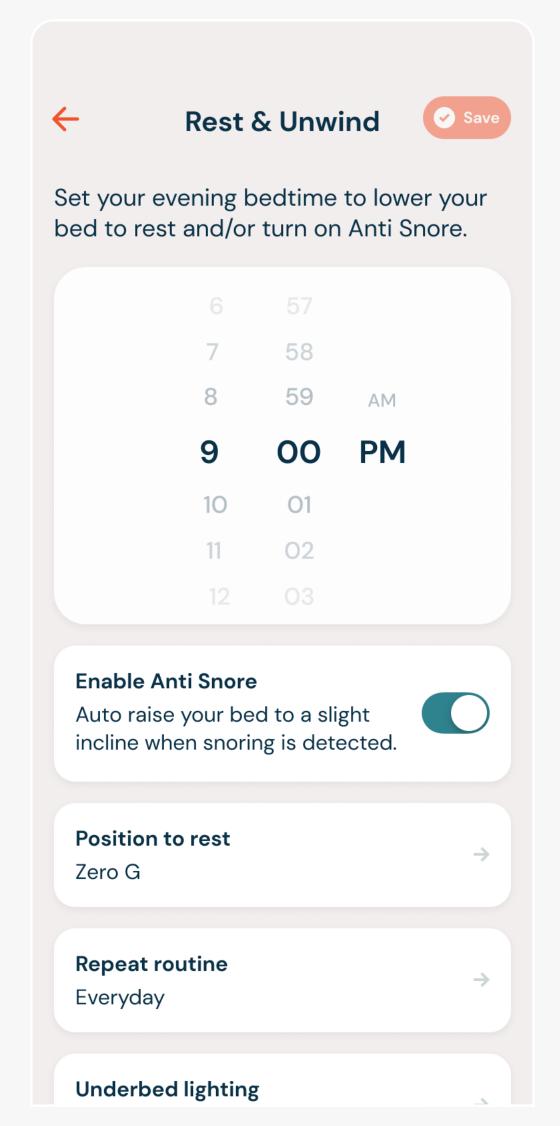

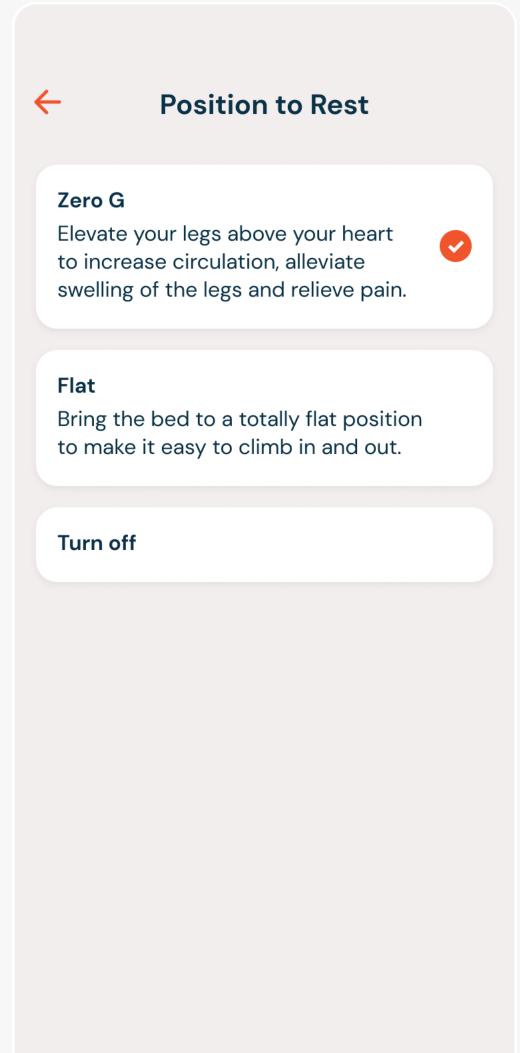



EDIT OR DELETE A ROUTINE

If you want to make changes or delete one of your routines, tap the routine card on the main page to open to the details.

Here you can make changes then select the "Save" button.

To delete a routine, tap on the button for "Delete Routine."

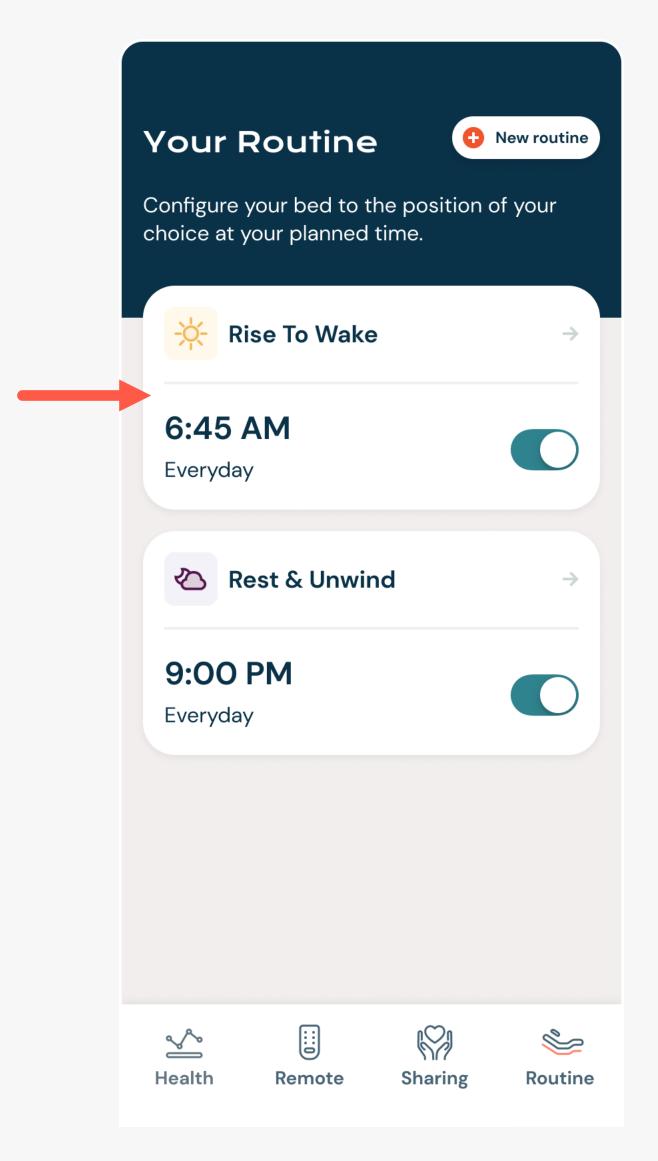

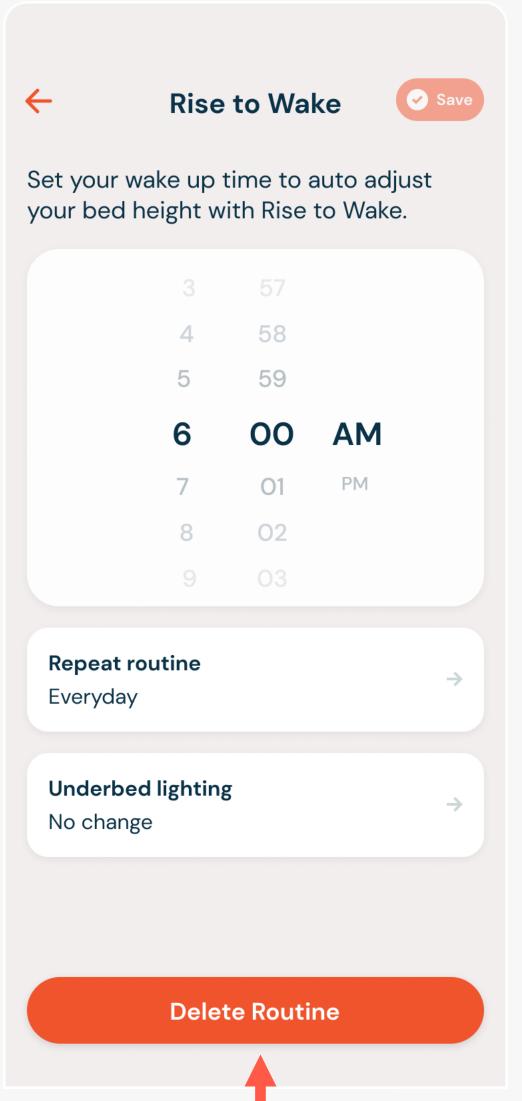

SHARING:
DAWN HOUSE
BED USERS


Your informative Dawn House Sleep
Reports can be shared with your loved
ones from the Sharing tab. Here you can
also view Sleep Reports shared with you.

SHARING:

FIRST ACCESS

As a Dawn House bed user, when you first access the Sharing tab you will be able to learn more about the benefits of sharing your health data.

You can then proceed to add a loved one to your share list, or send a one-time health report, but only if you so wish to.

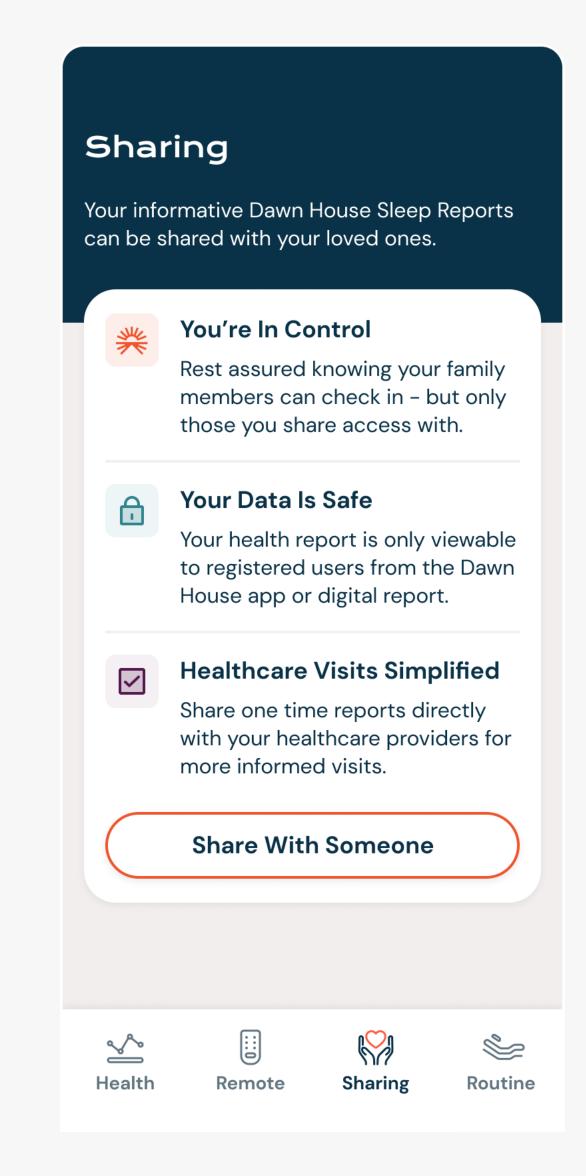

SHARING:


ADD TO MY

SHARE LIST

When you add to your share list, your loved one will receive an invite to access your sleep reports. They will have their own Dawn House account login and be able to view your daily, weekly and monthly metrics.

To add someone to your share list select the button for "Add to my share list" and enter their email address(es).


You will see the emails of anyone you have invited to view your report on the default My Share List toggle view on the Sharing tab.

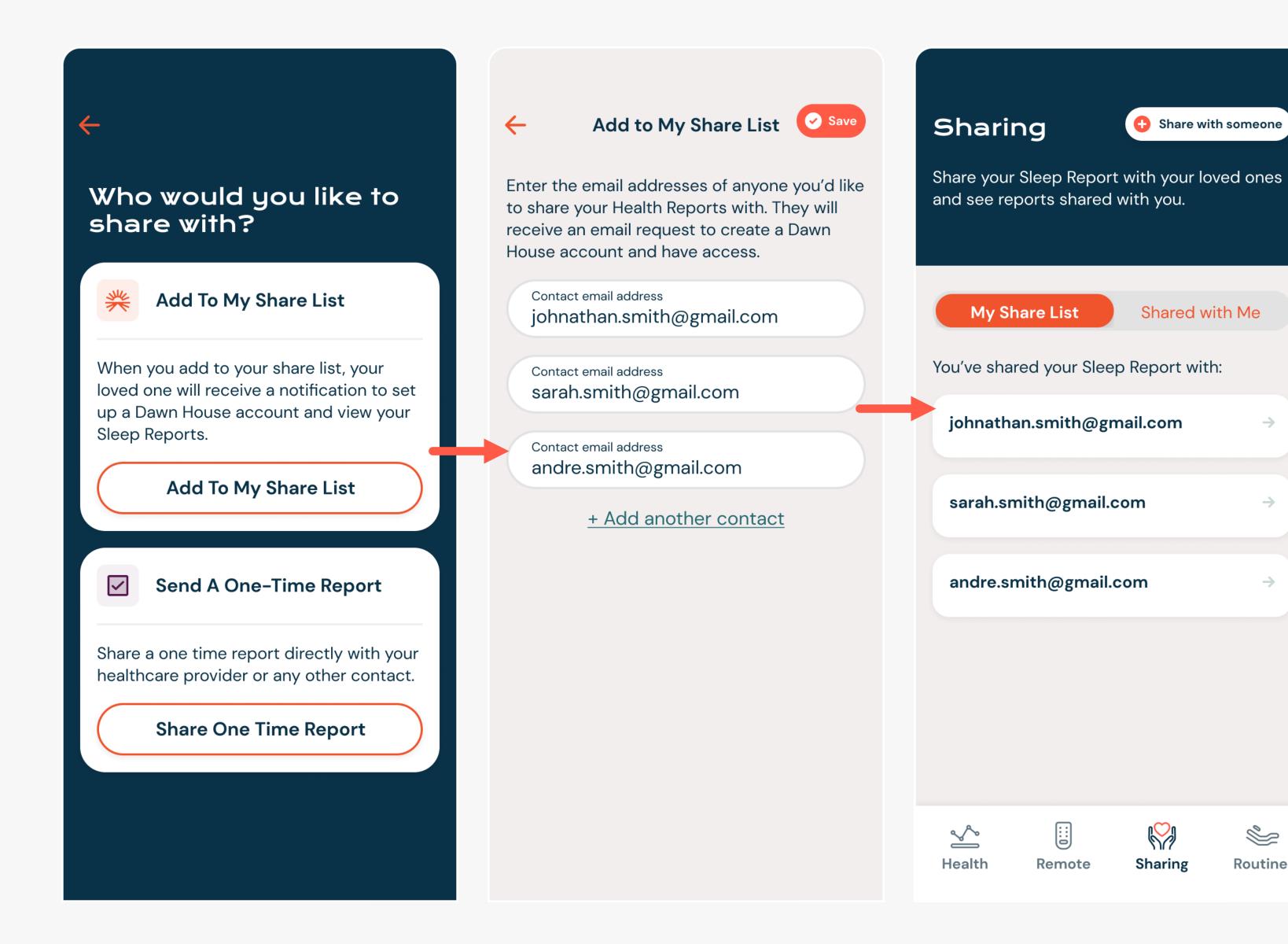

SHARING: SEND A ONE-TIME HEALTH REPORT

You can share a one-time health report via email to your healthcare providers or any other contact you wish.

Select the button for "Share one time report" then enter the email address of the person you'd like to share with.

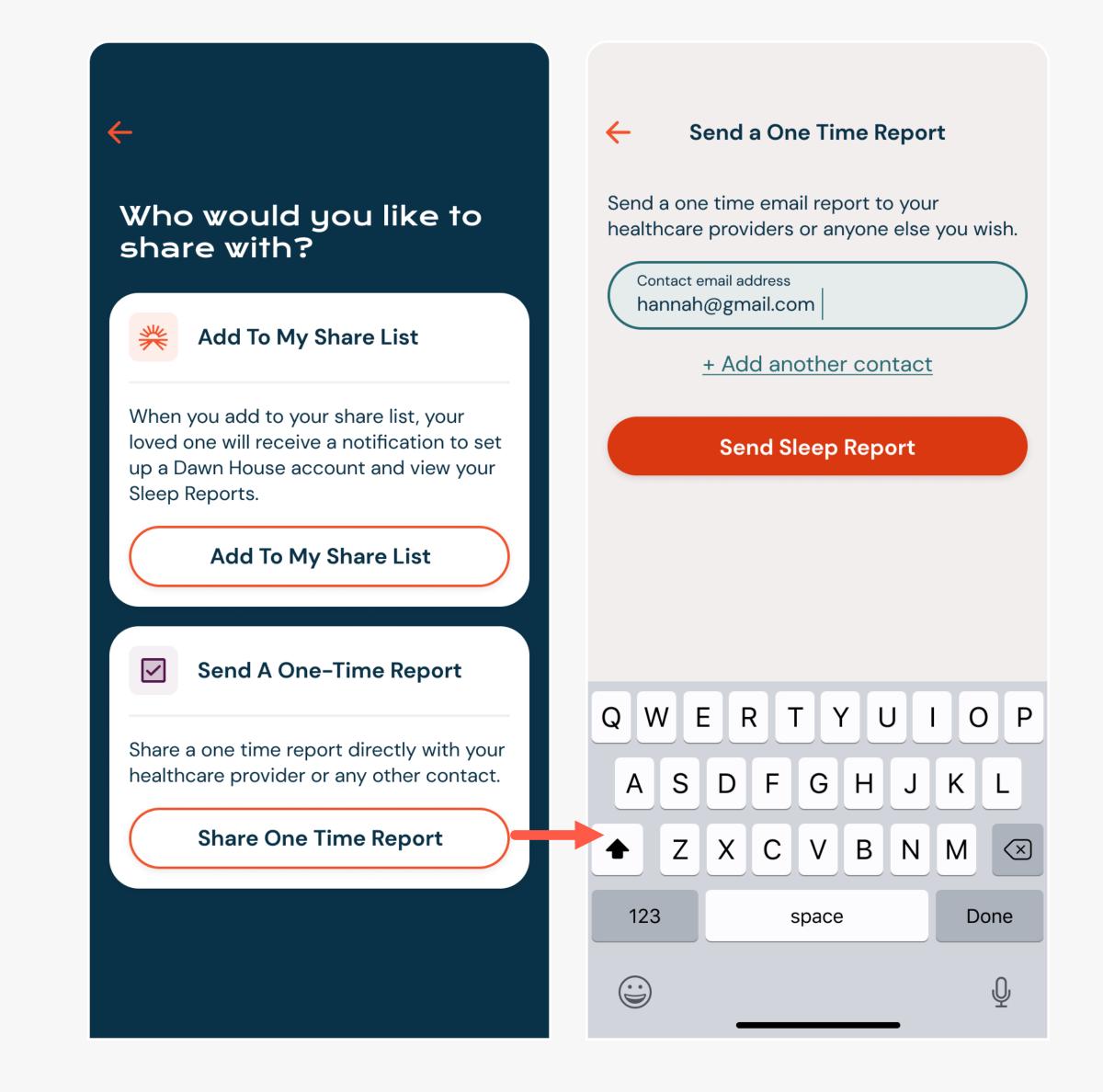

SHARING:


MY SHARE LIST & SHARED WITH ME


When you've added someone to your Share List or a Sleep Report has been shared with you, you will see the tab display.

To update a contact on your Share List, select the email address and proceed to edit or remove the contact on the next page.

When you tap "Shared with Me" you will see the Dawn House sleep reports shared with you.

At any time you can select the top right button "Share with Someone" to add a loved one to your share list or to send a one-time health report.

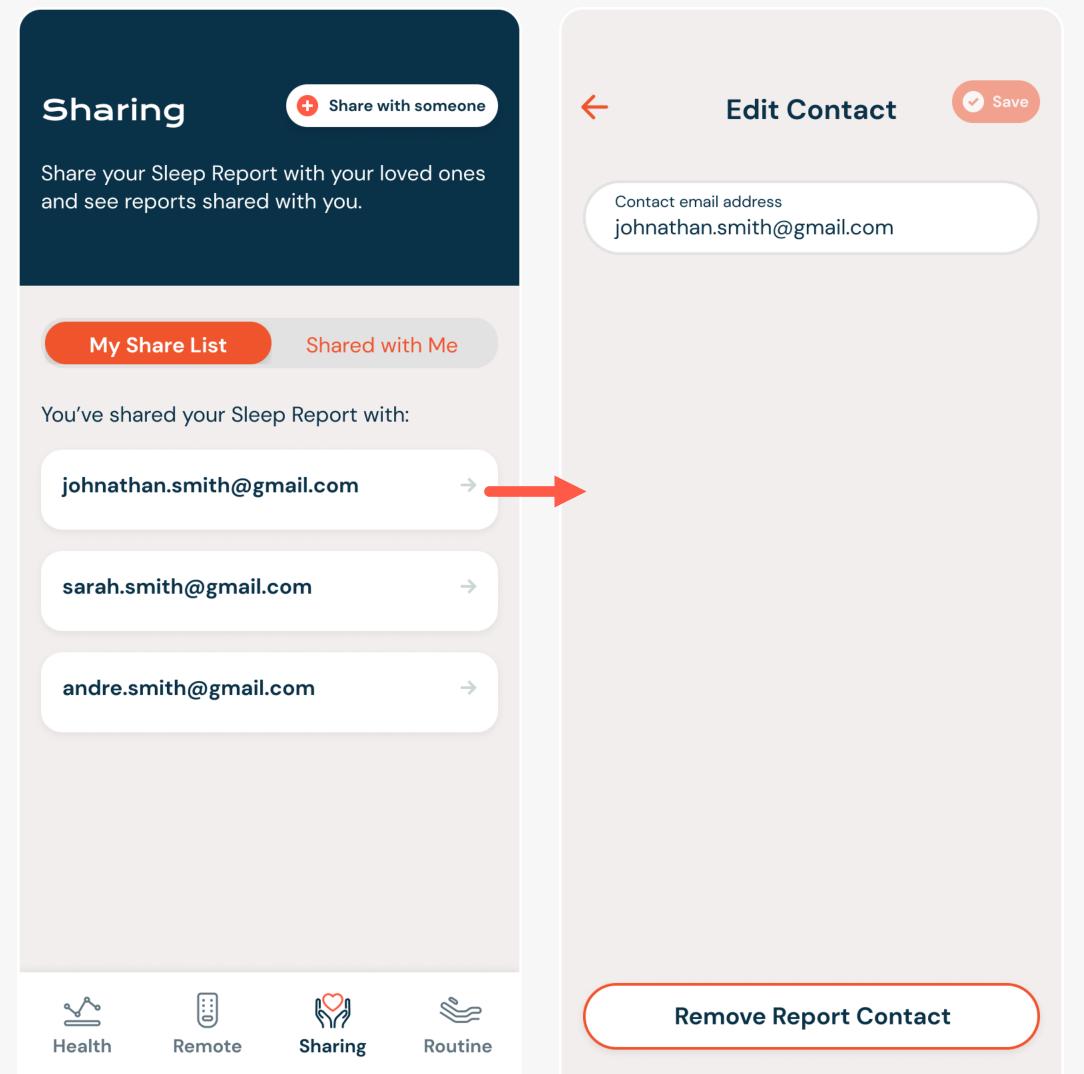

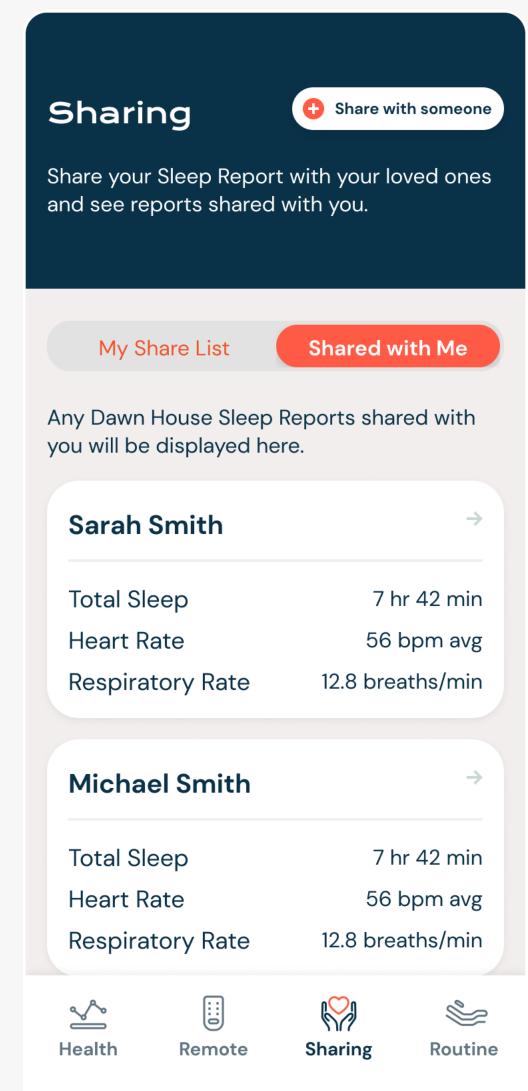

SHARING: FAMILY ACCOUNTS

When you create a Dawn House account to view your loved one's Sleep Report, your viewing experience will be simplified as compared to a Dawn House bed user. Sharing will be your default home screen.


SHARING:


HOME SCREEN


Once you create your account, if a loved one has already shared their Sleep Report your home screen will reflect their high level metrics. Tap the card to view the full Sleep Report.

If there haven't been any Sleep Reports shared with you, you will see a prompt to help your loved one initiate sharing from their Dawn House app.

To see your App Settings, select the top right icon from the main Sharing screen.

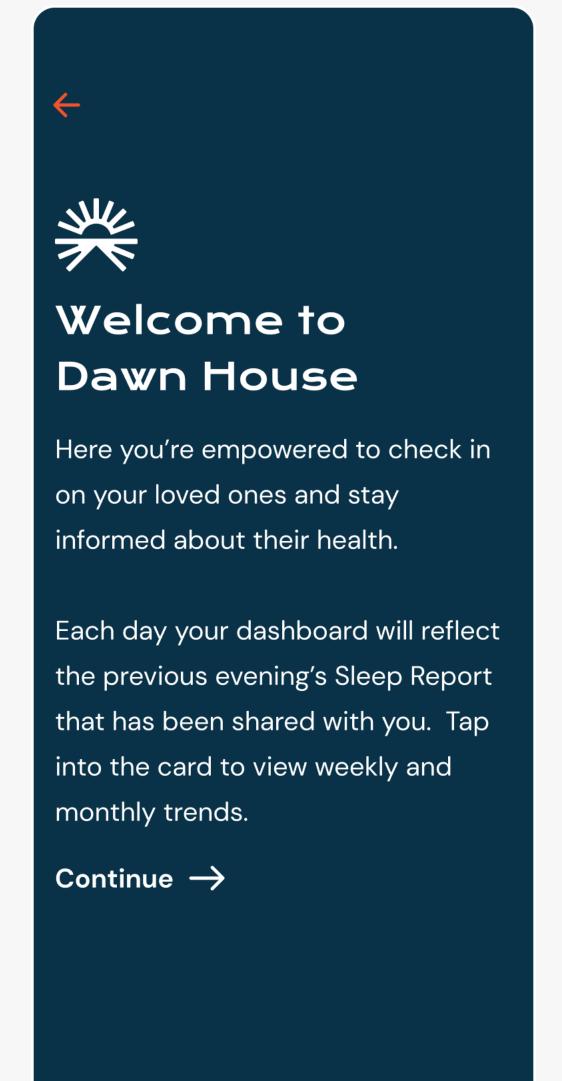

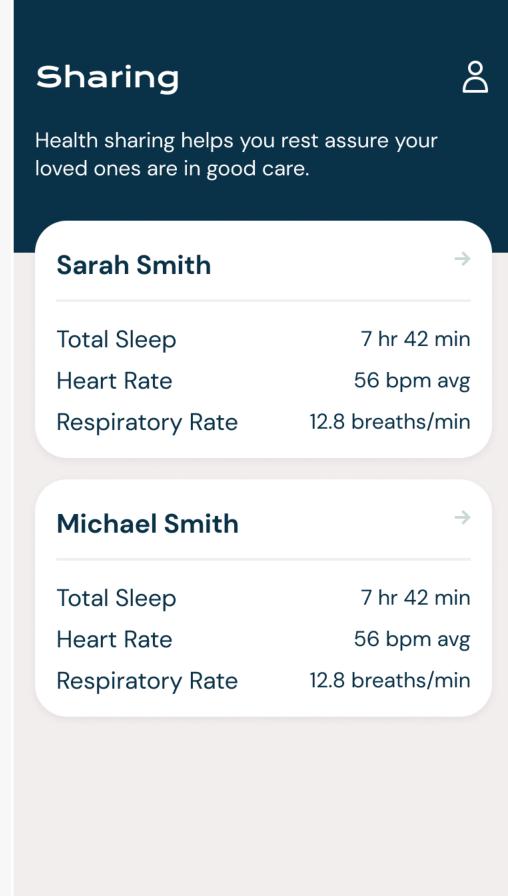

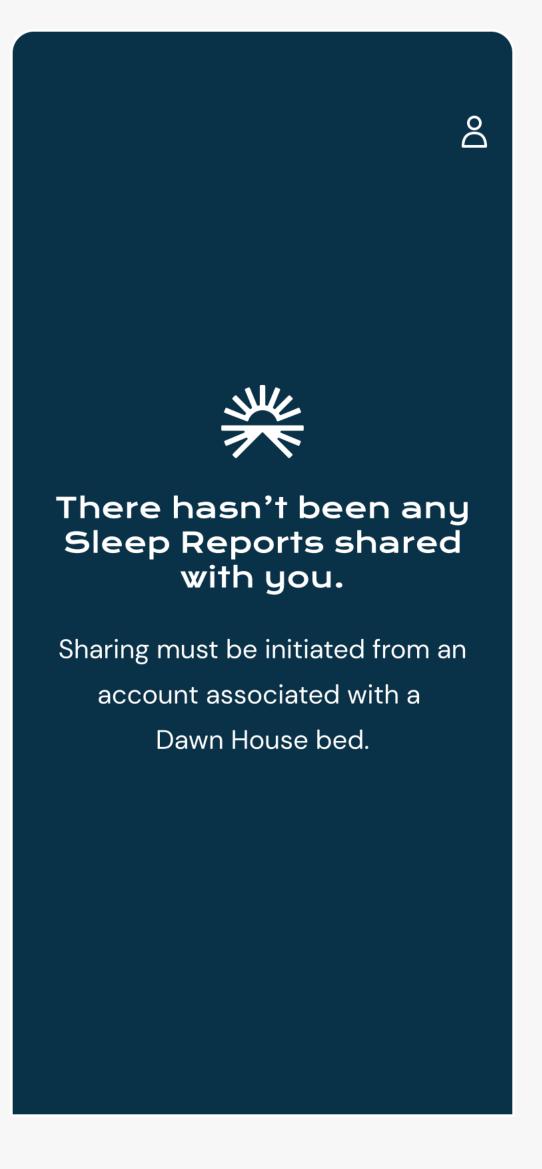



VIEWING A SLEEP REPORT

When you open a Sleep Report, you will see your loved one's name prominently displayed.

Tap the calendar icon to change the date you are viewing health metrics for.

Tap into any card to view the detailed metrics based on the previous evening, week and month.

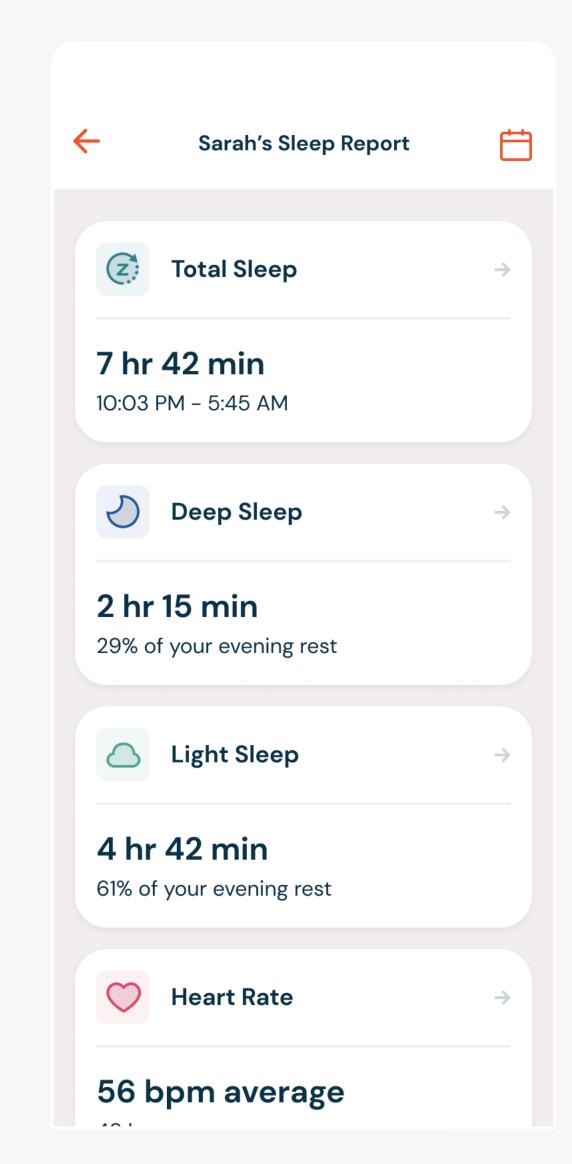

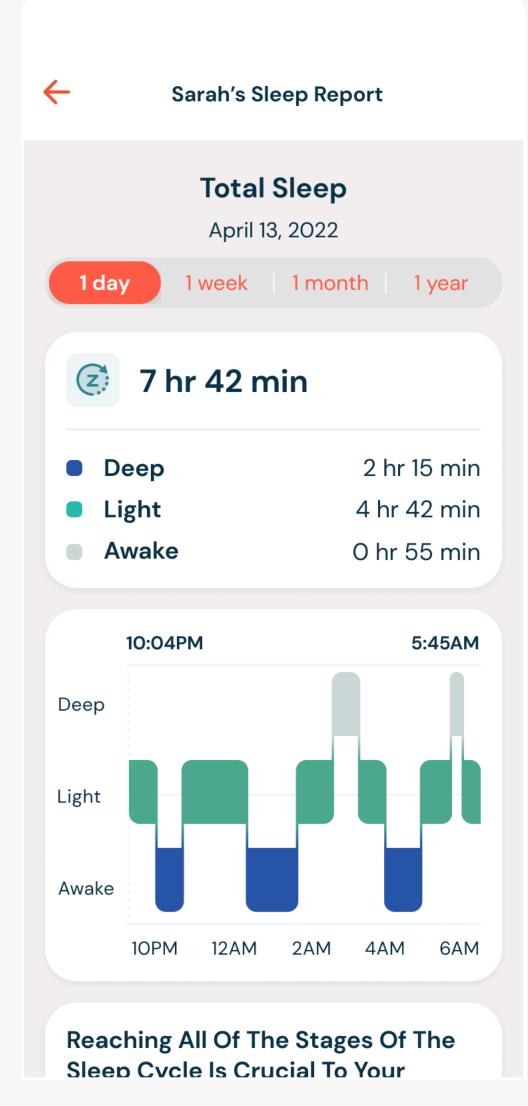

APP SETTINGS


As a Dawn House user with a Family Account, your App Settings will be simplified.

To change your password, follow the instructions on Page 47 of this manual.

To update your profile settings, see Page 49.

To change your FaceID settings, see Page 54.

If you purchase your own Dawn House bed, you can proceed to connect your bed from the App Settings page. See page 10 for bed connection details.

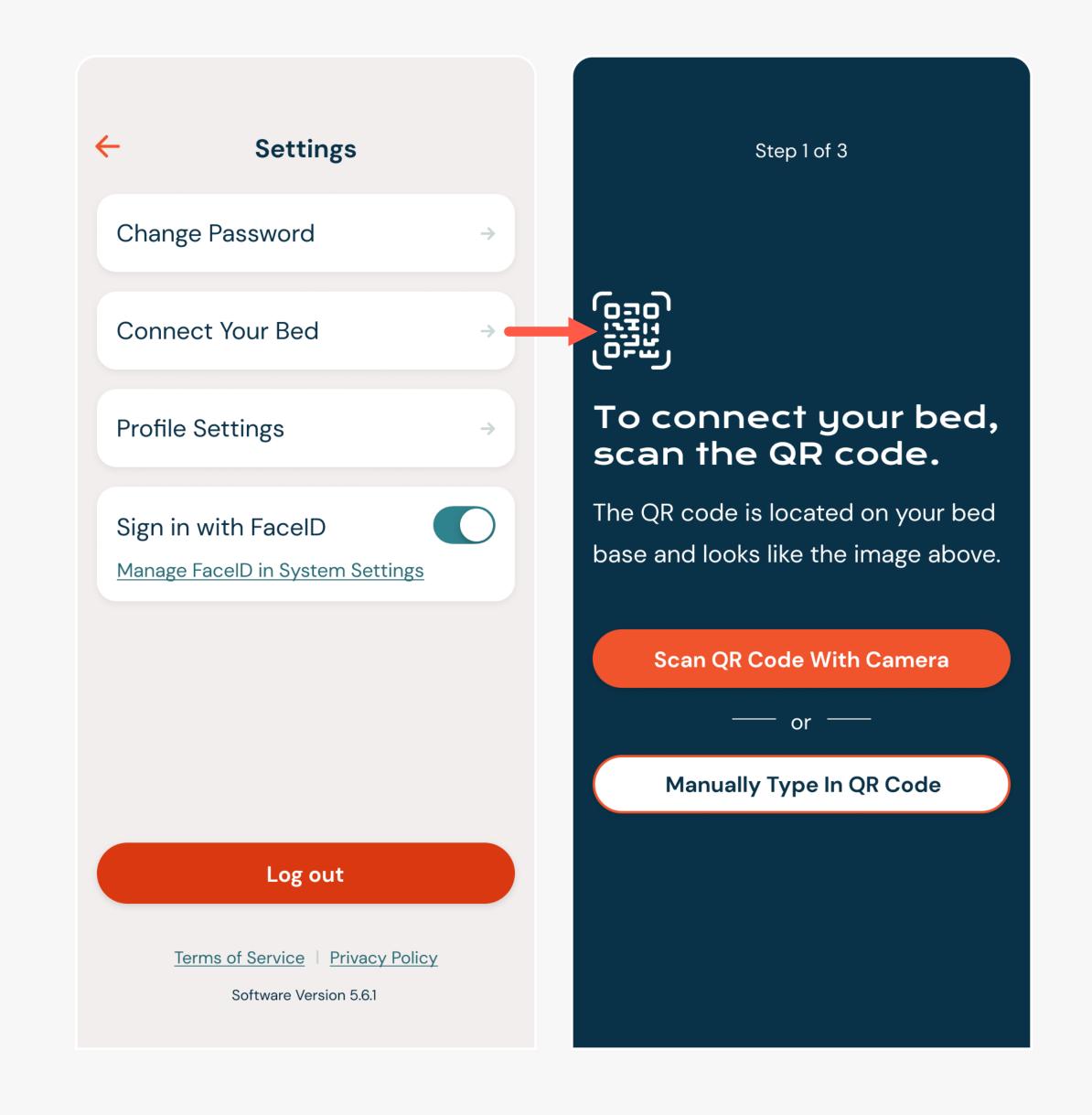

SIGN IN & FACEID

THERE ARE SEVERAL WAYS TO SIGN IN:


- 1. Email and password
- 2. Facebook, Google or Apple
- 3. FaceID (iOS users only)


SIGN IN WITH EMAIL ADDRESS


When you open the Dawn House app, use your email and password to sign in.

If you check the box for "Remember Me" you will stay logged in for your future visits and skip the sign in process.

If you have recently updated your app, you may see the Create Account screen. Click the link "Already have an account? Sign In" to be taken to the main Sign In screen.

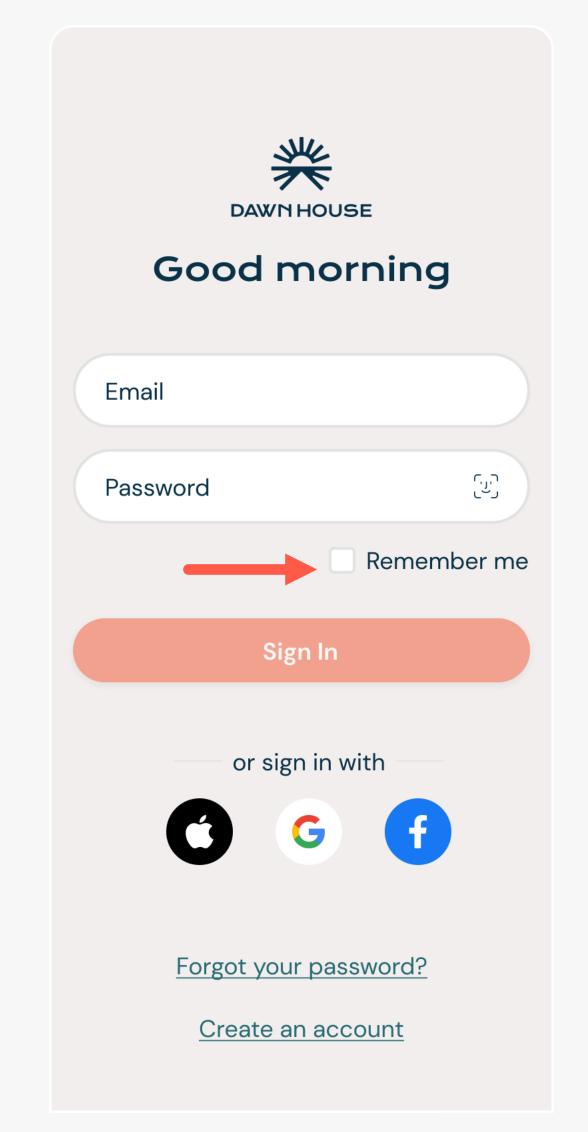

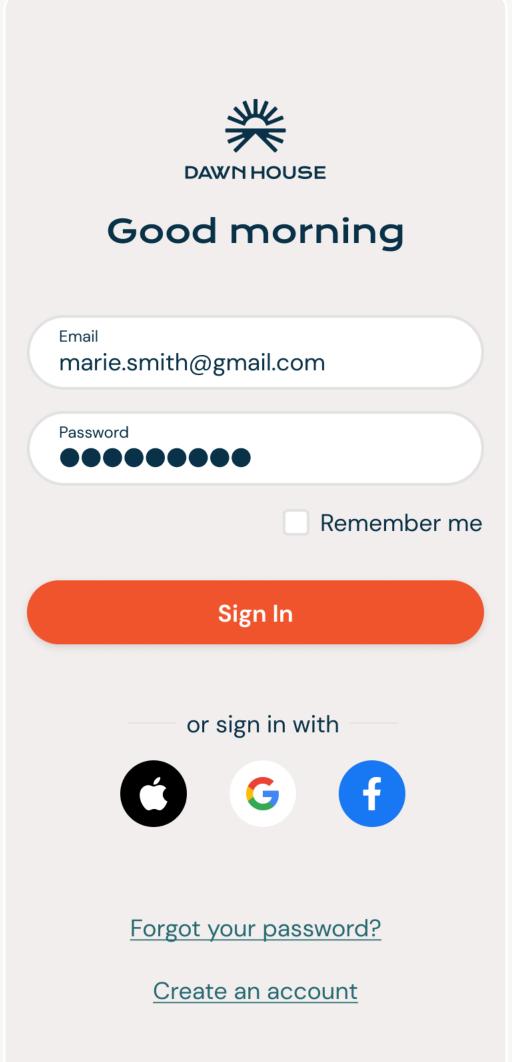

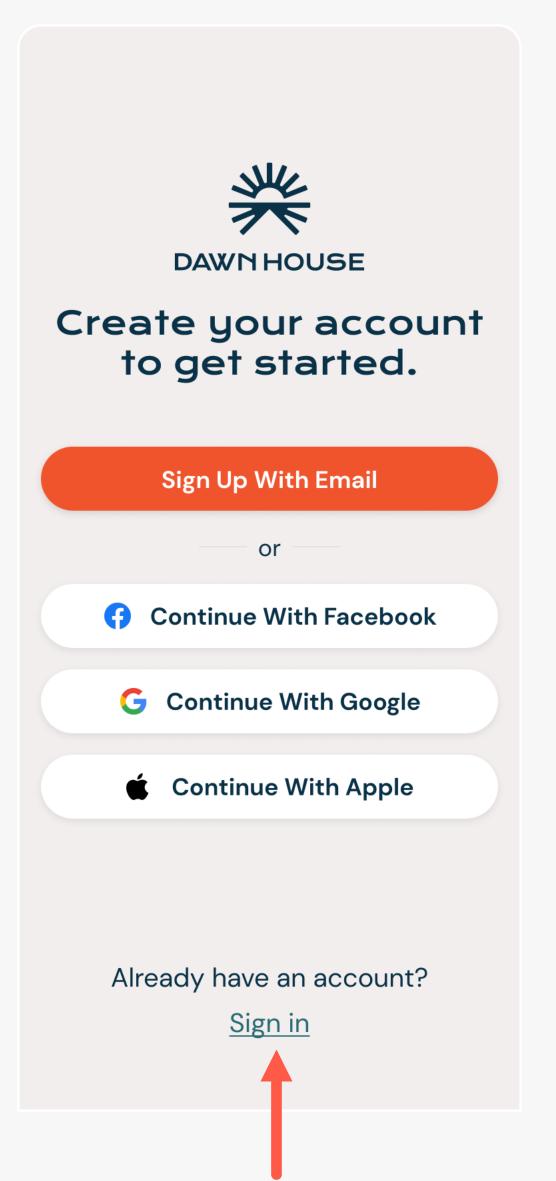



SIGN IN WITH GOOGLE, FACEBOOK OR APPLE

When you open the Dawn House app, you can select the icon to sign in with Facebook, Google or Apple.

You will be automatically directed to your home page after your account has been authenticated.

If you have recently updated your app, you may see the Create Account screen. Click the link "Already have an account? Sign in" to be taken to the main Sign In screen.

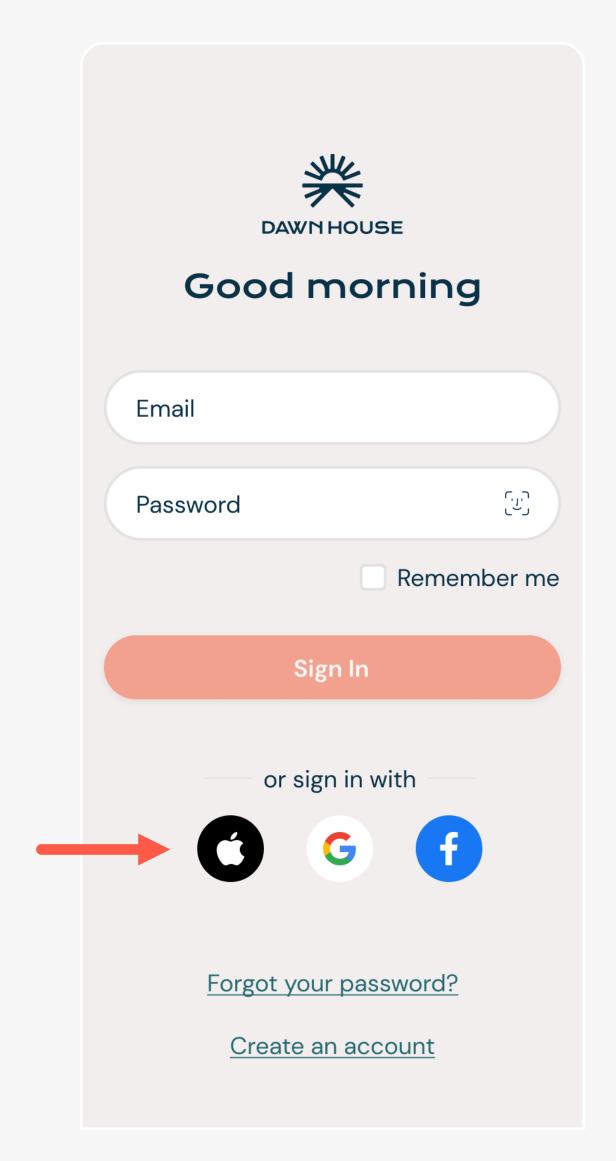

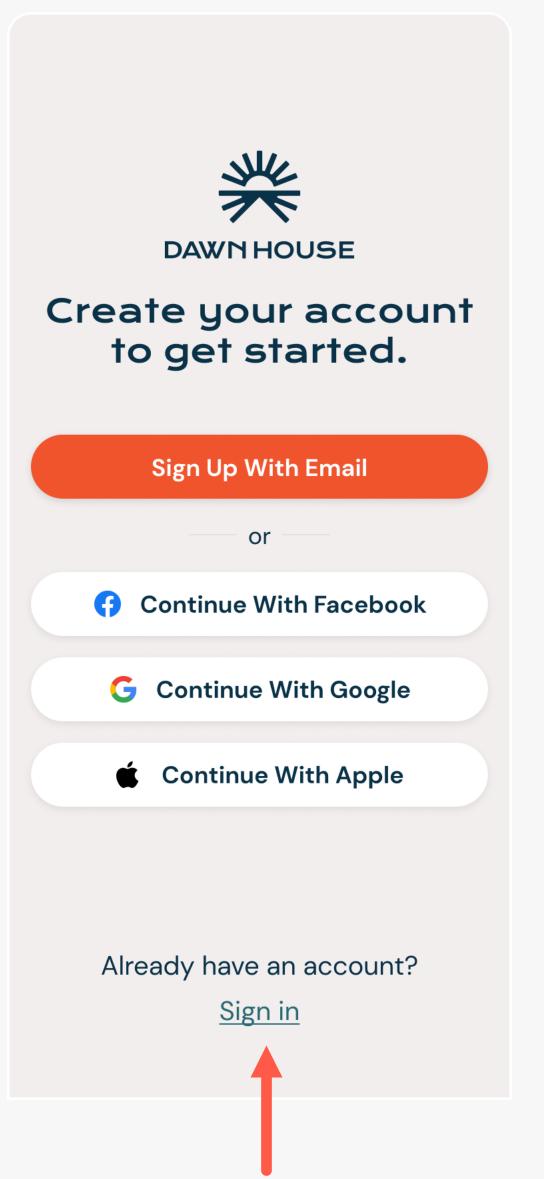



SIGN IN WITH FACEID


You can also choose to sign in with FaceID. Select the link "Use FaceID to Sign In."

It will open the FaceID prompt where you can grant permission and sign in.

On future visits when you have enabled FaceID, you will be automatically logged in.

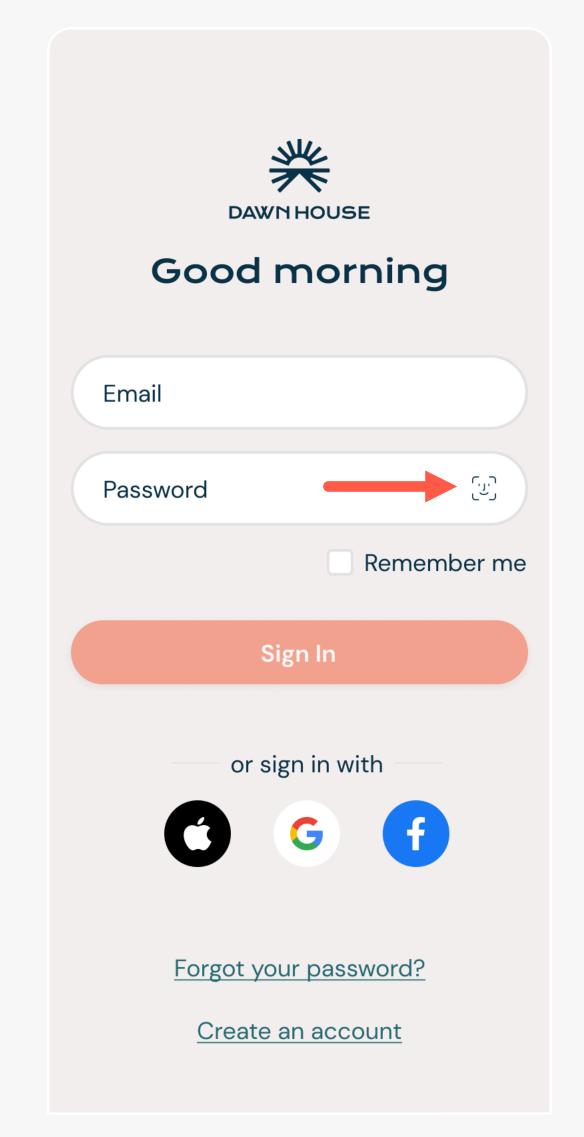

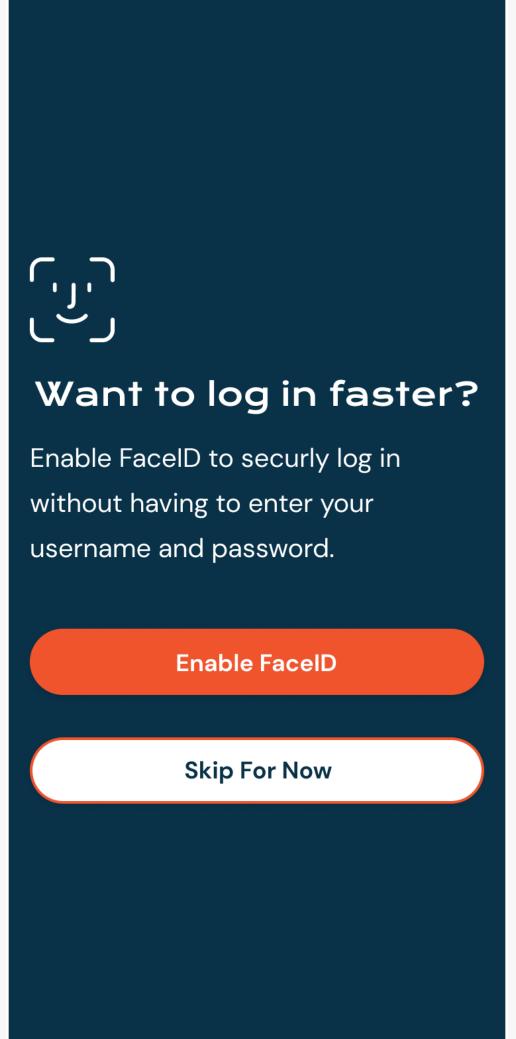

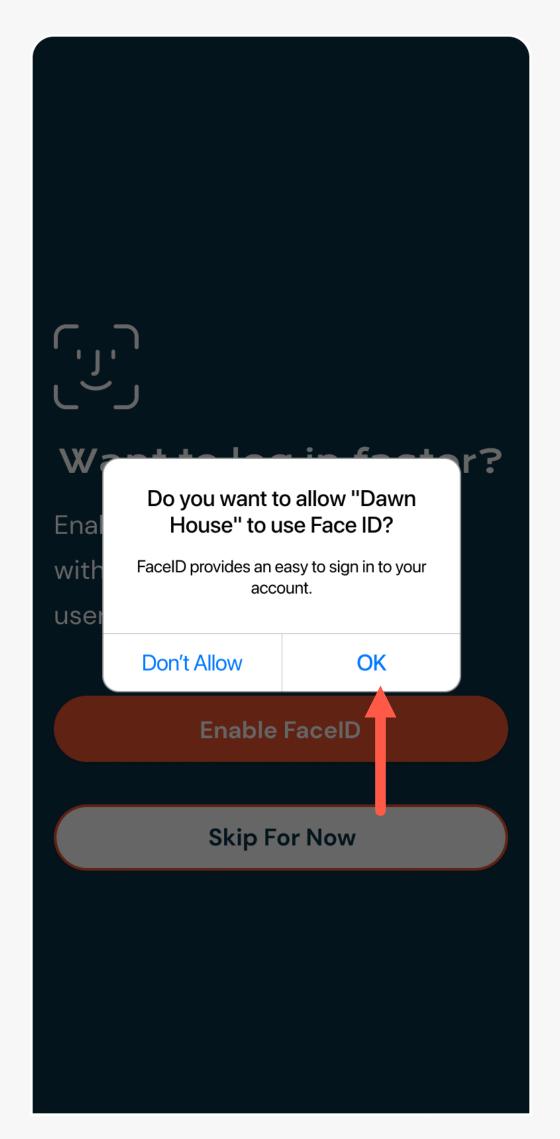


RESET YOUR PASSWORD


Your password is reset through your email address. If you are having trouble, please contact info@dawnhouseliving.com.

RESET YOUR PASSWORD

- 1. Click the "Forgot your password?" link to open to start the forgot password process.
- 2. Enter your email address and Dawn House will send a verification code to your email.
- 3. Enter the verification code and you will be prompted to create a new password. Your new password must be at least 8 characters and include any special character (ex. !@#\$%).

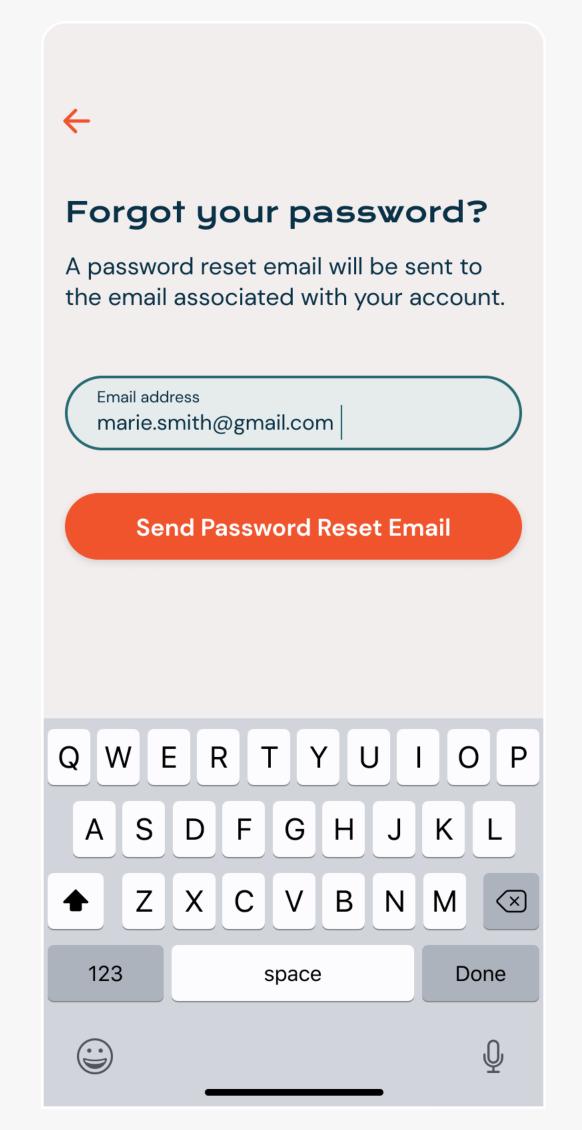



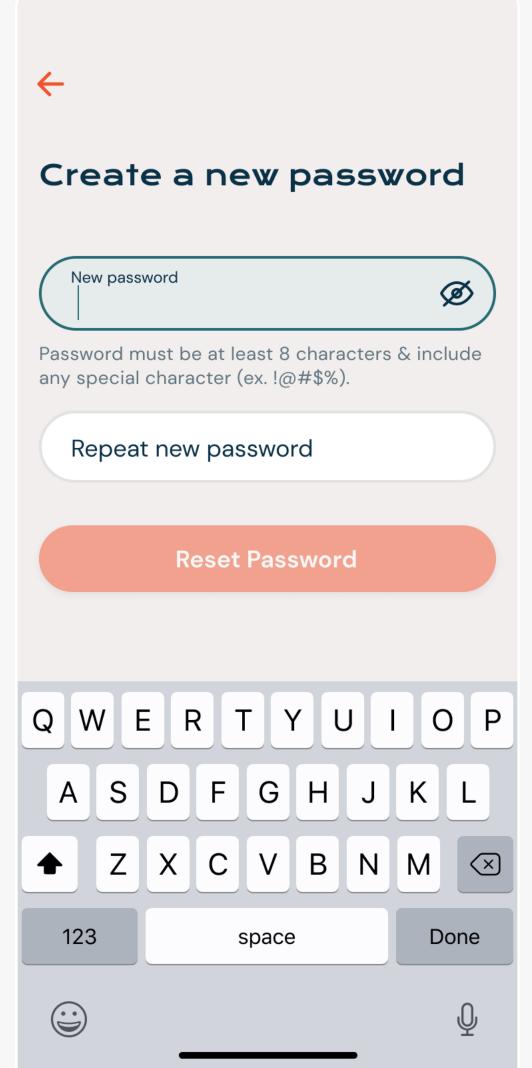


ACCOUNT & APP SETTINGS


You can make changes to your profile, review your bed connection and share your Sleep Report all from the Account Settings screen.

OPEN YOUR APP SETTINGS

Click the Settings gear icon on the home page to open the App Settings.

When your Dawn House bed is connected, you will see the full list of Settings. When your bed has not been connected, you will see a simplified list.

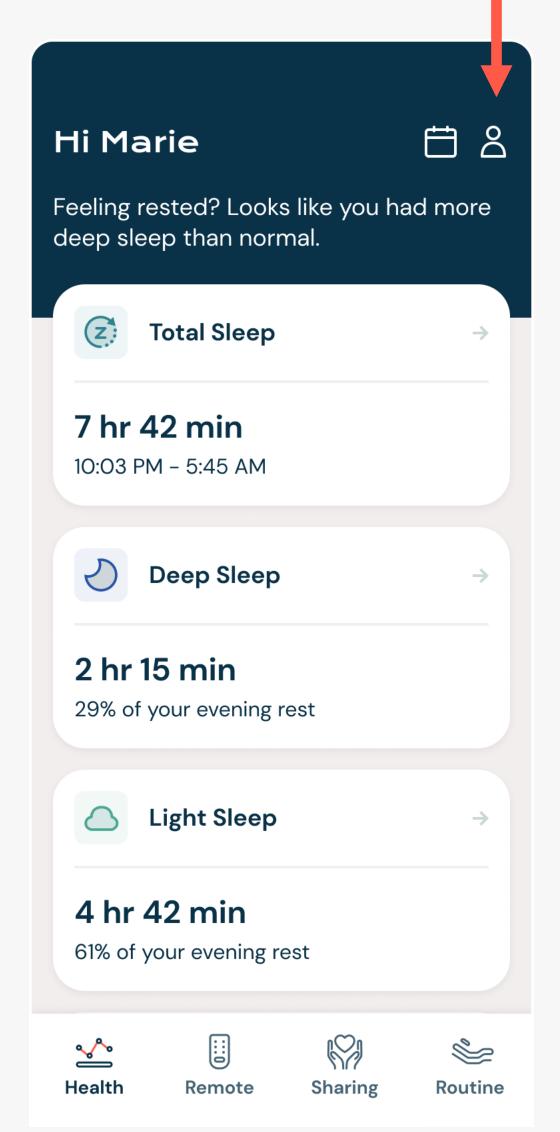

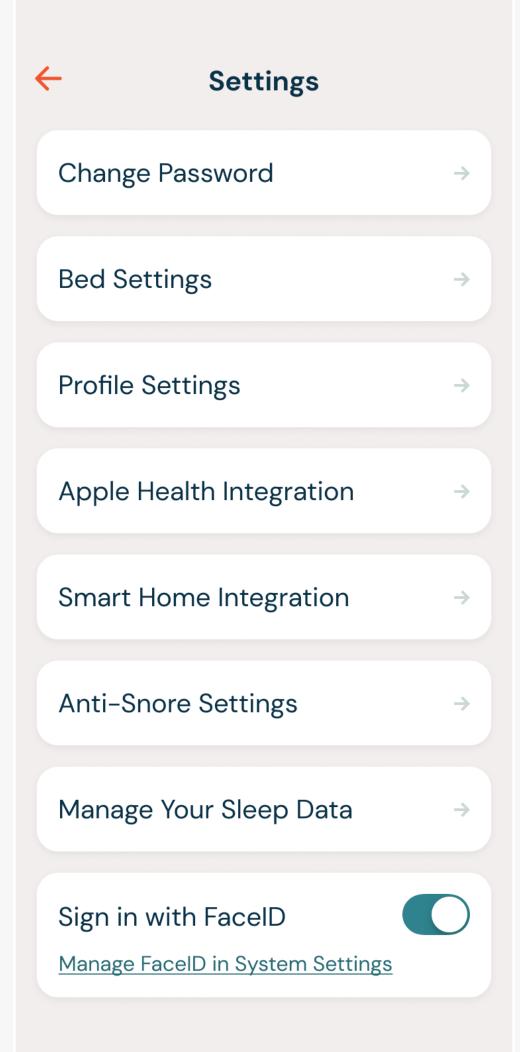


CHANGE YOUR PASSWORD


To change your password, tap the "Change Password" tile.

You will be prompted to enter your current password then provide a new password.

Your new password must be at least 8 characters and include any special character (ex. !@#\$%).

If you are having trouble remembering your password, select the "Change with a reset email link."

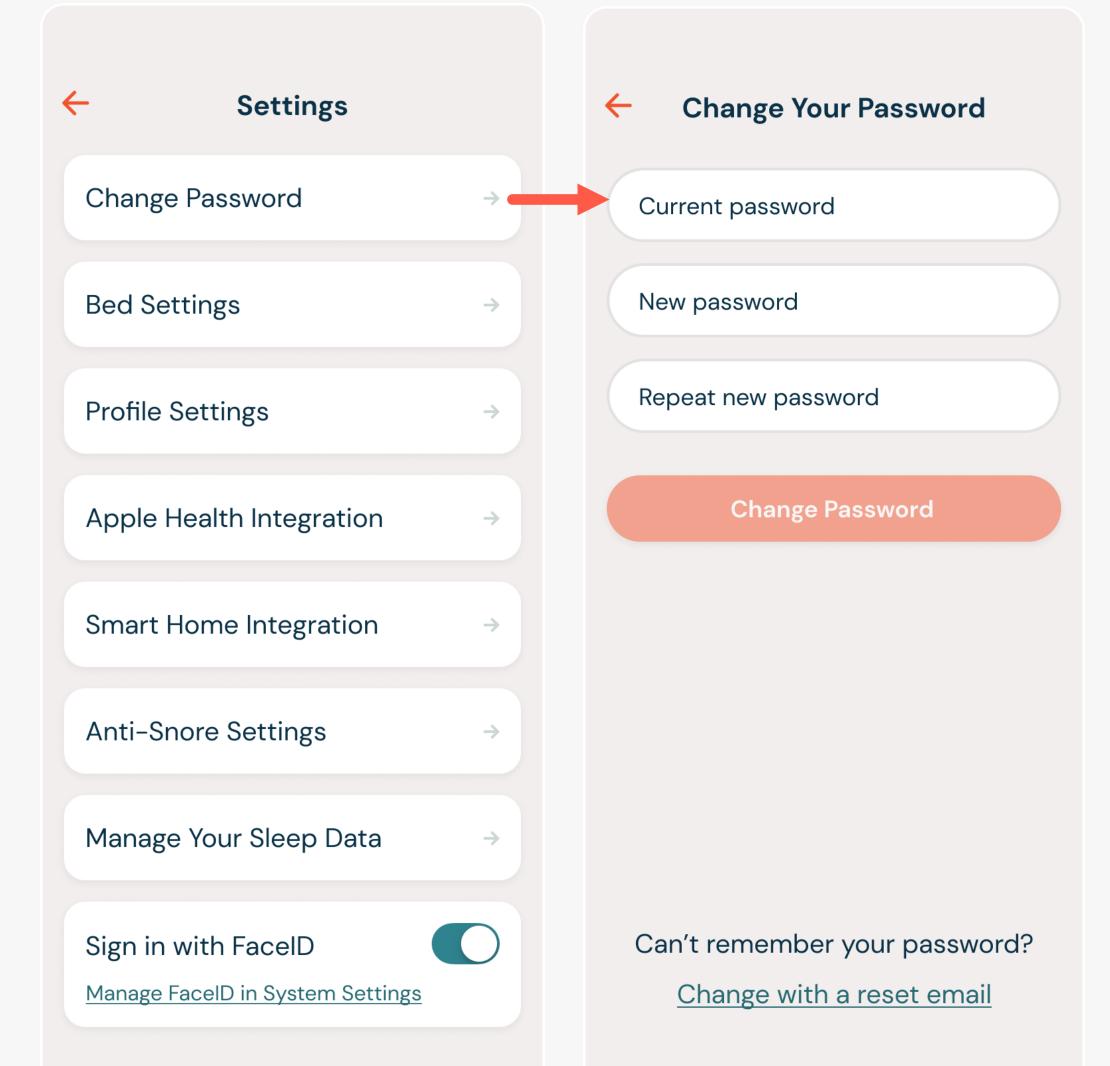

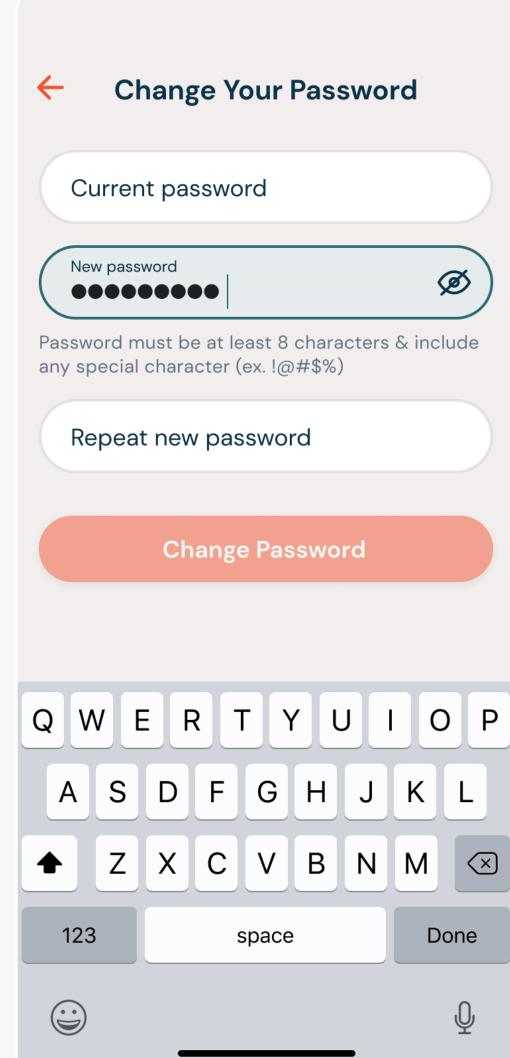

Bed Settings

No, keep my bed connected

Yes, disconnect my bed


YOUR BED SETTINGS

The Bed Settings page displays details about the Dawn House bed your account is connected to.

Here you will find the device number, online or offline status and the bed type.

If you want to disconnect your bed from the app, select the link "Disconnect Bed." You will see a confirmation.

You will be able to see historical data but you must reconnect your bed to see any new health data.

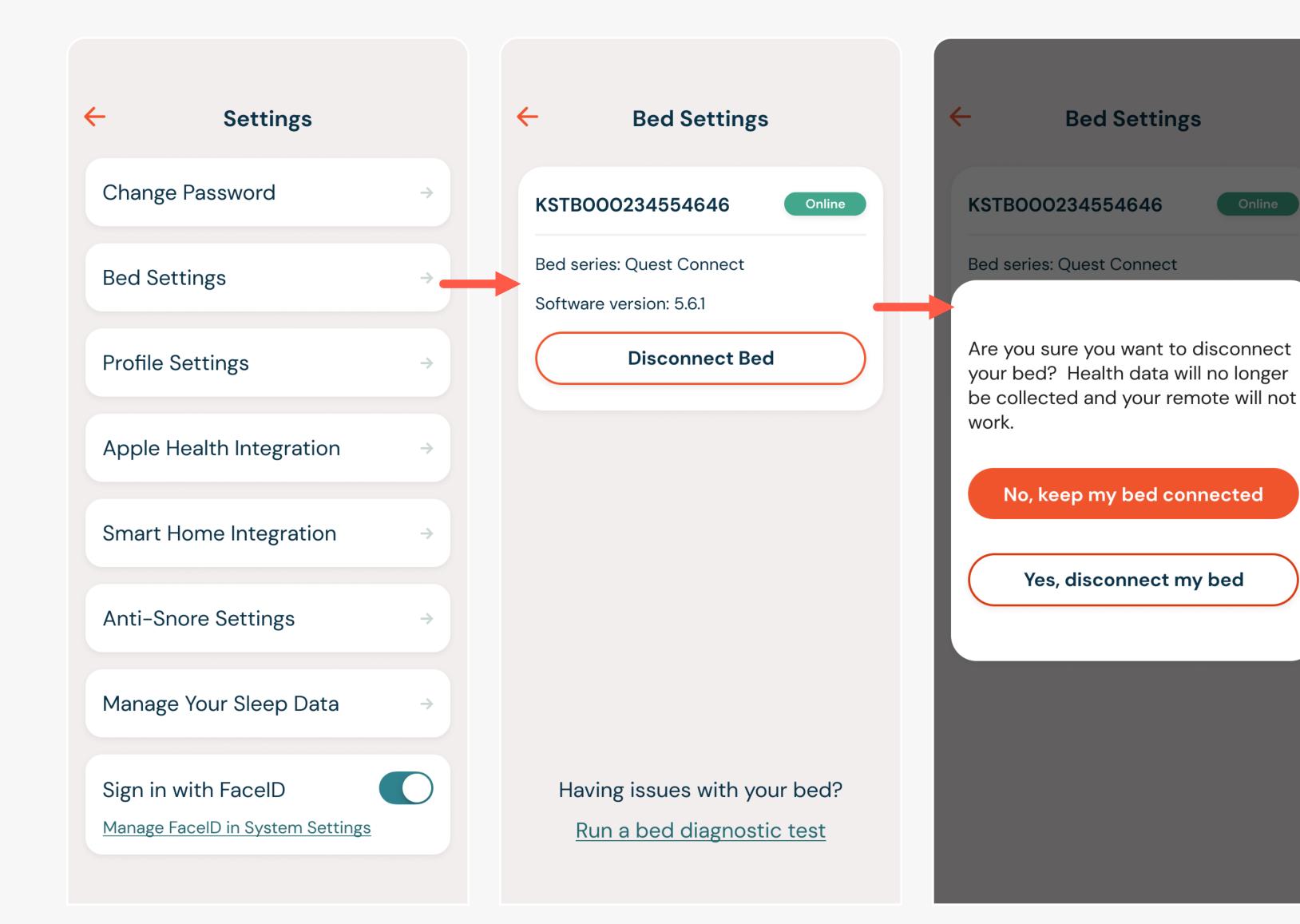


YOUR PROFILE SETTINGS

Your Profile Settings includes your basic personal information. You can update your name, mattress thickness and weight.

If you have a queen size Dawn House bed it is essential that the app reflects the side of the bed you currently sleep on. Select the form field for "Bed Side" to open the dropdown and change your side. If you have any other size bed, you will not see this menu option.


If you tap "Delete Your Account", you will be redirected to the Dawn House website to remove your account.

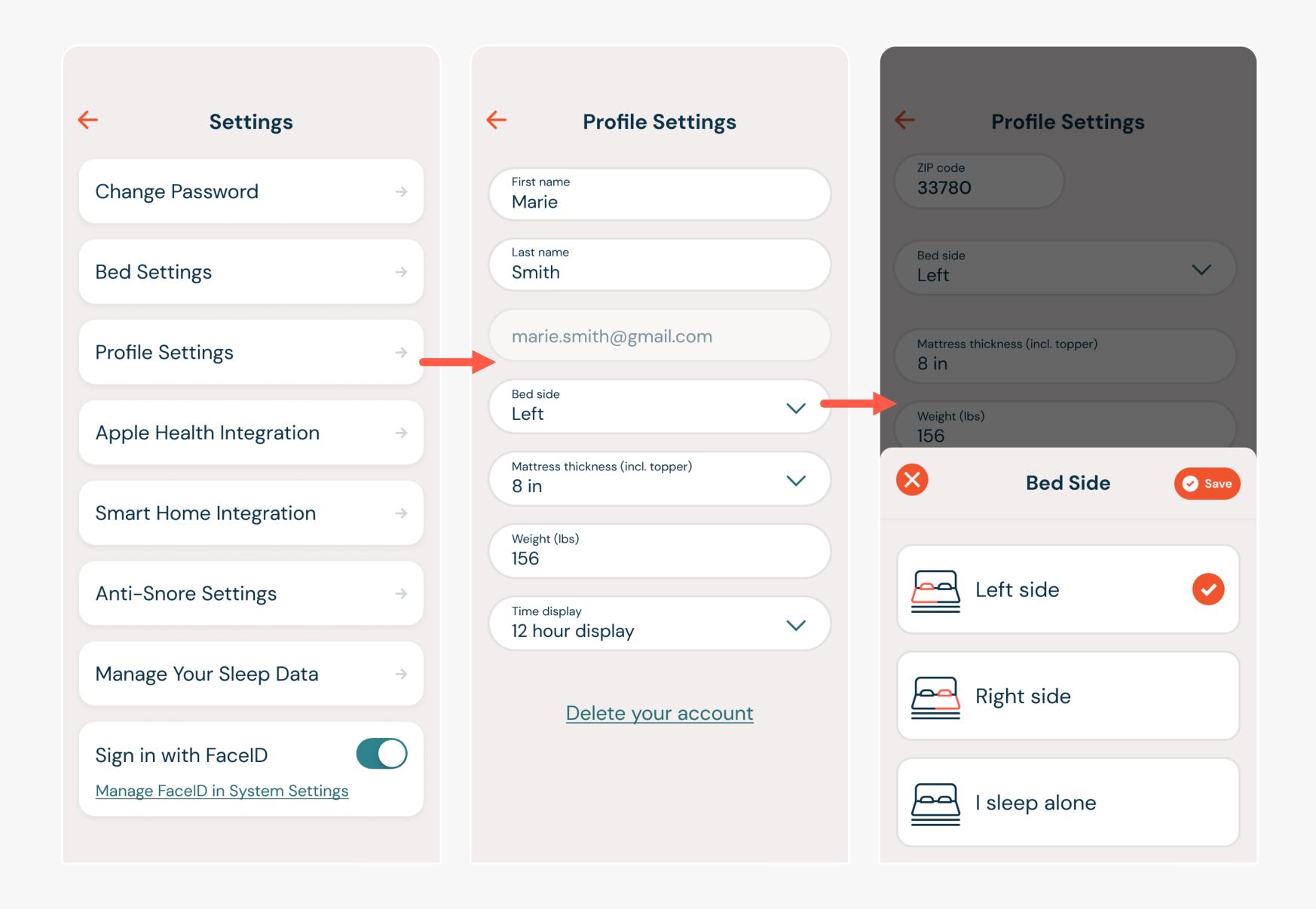


ACCOUNT & APP SETTINGS 50

APPLE HEALTH INTEGRATION

Lorem ipsum

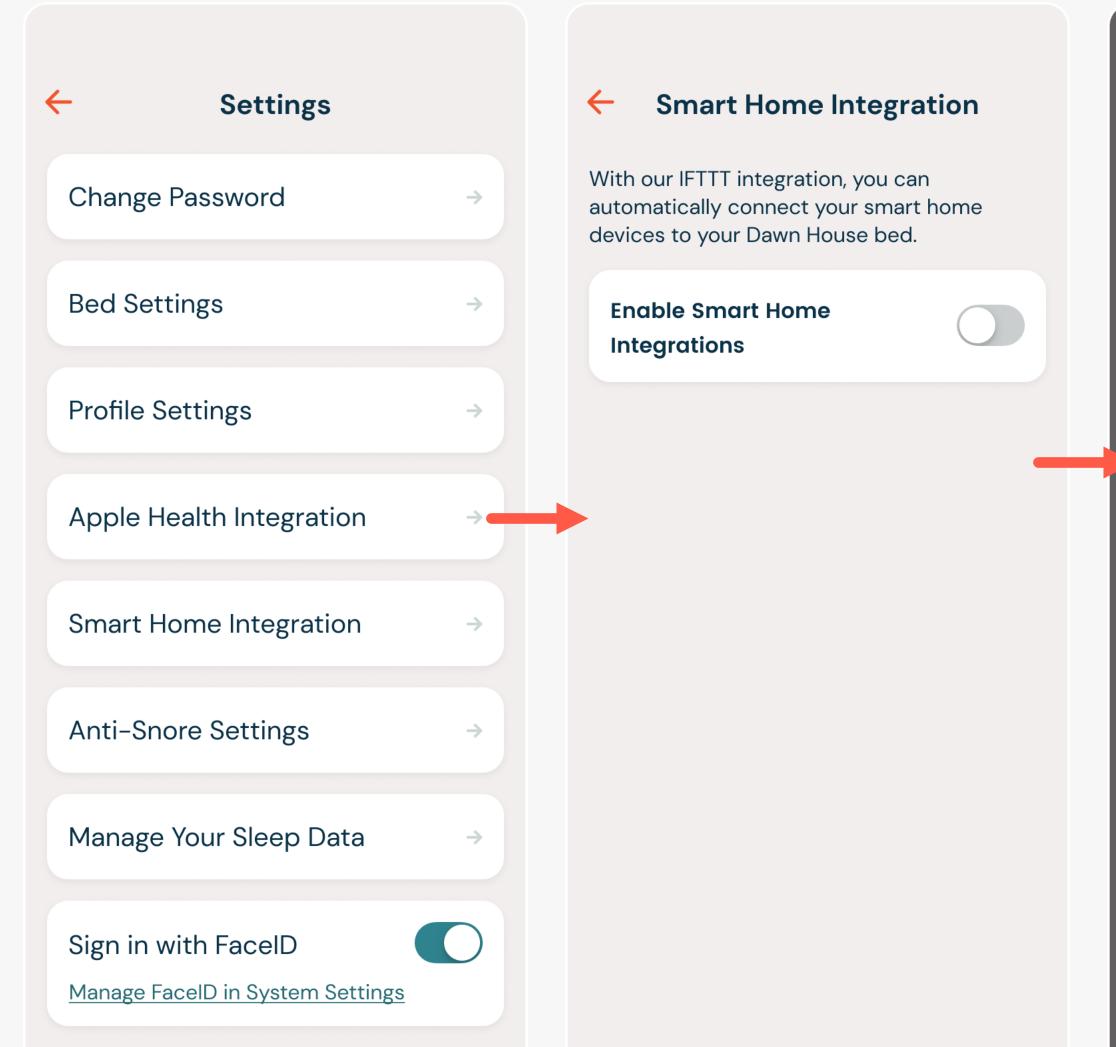


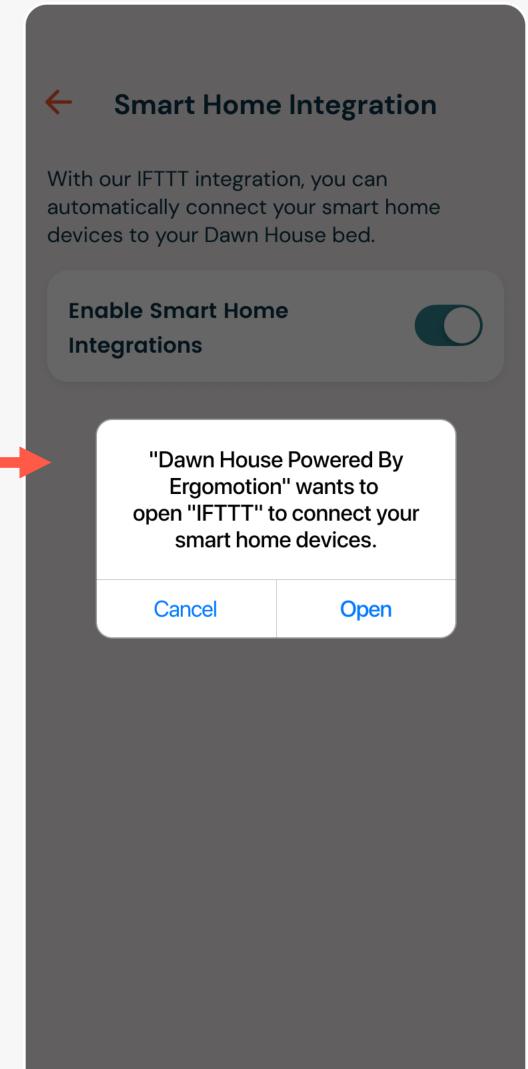

51 ACCOUNT & APP SETTINGS

GOOGLE HEALTH CONNECT INTEGRATION

Lorem ipsum

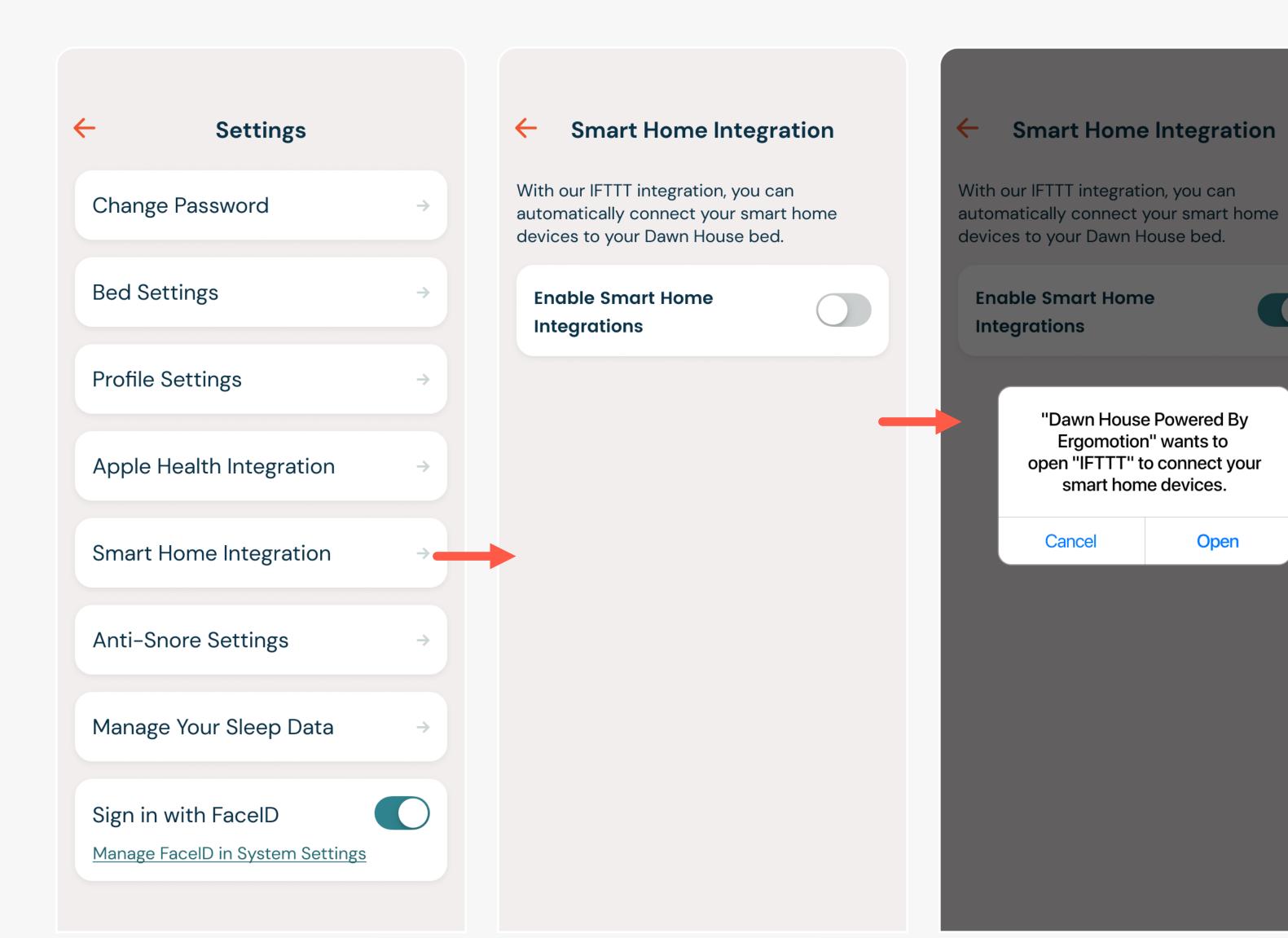

Smart Home Integration

"Dawn House Powered By

Ergomotion" wants to

open "IFTTT" to connect your

smart home devices.

Cancel

Open

Smart Home Integration

"Dawn House Powered By

Ergomotion" wants to

open "IFTTT" to connect your

smart home devices.

Open

With our IFTTT integration, you can

devices to your Dawn House bed.

Enable Smart Home

Cancel

Integrations

automatically connect your smart home


SMART HOME INTEGRATION

The Smart Home interface provides a way to connect smart home devices. You will be asked to create an IFTTT account.

Download the IFTTT app for more integrations:

iOS Devices


Android Devices

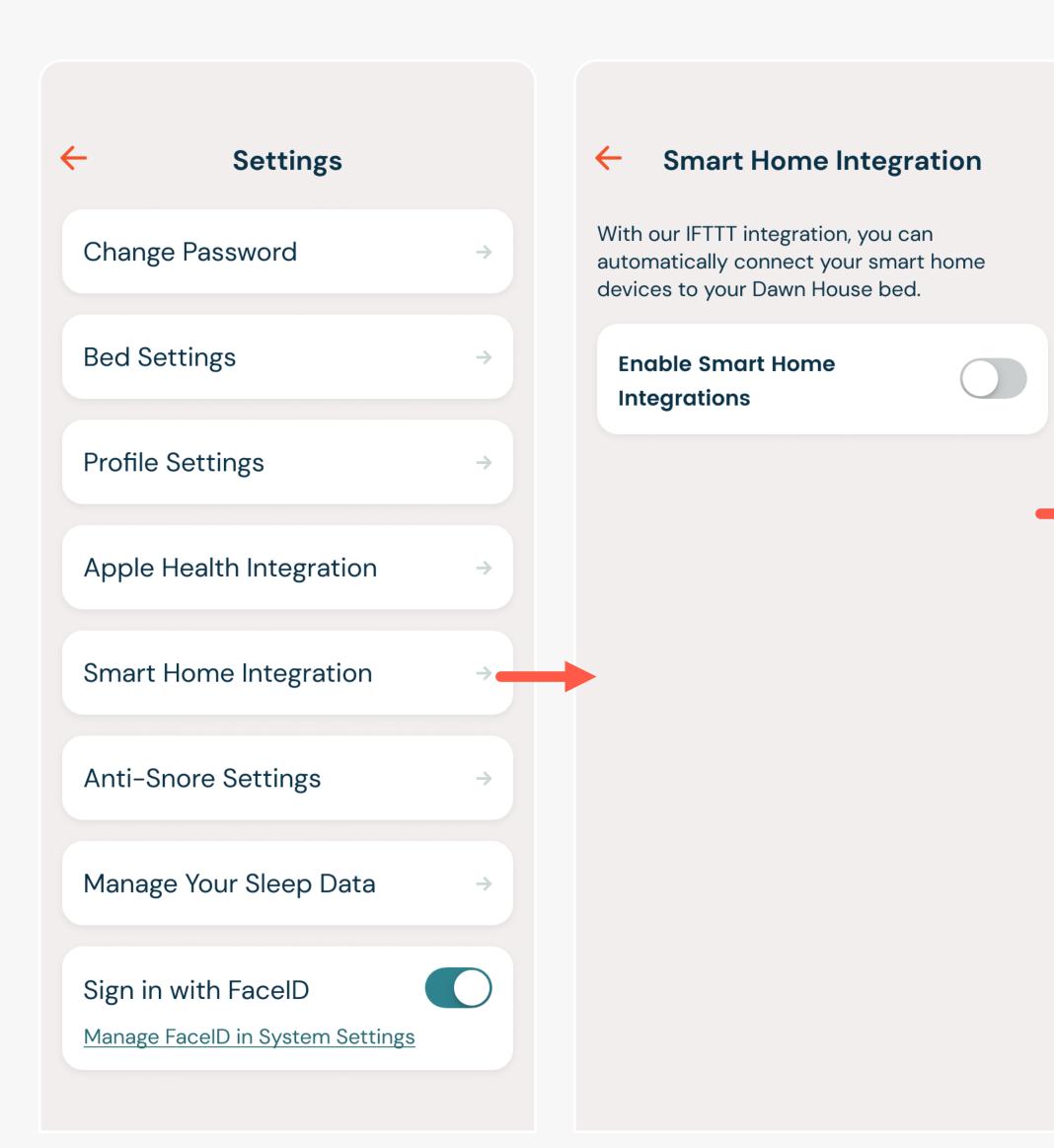

ANTI-SNORE SETTINGS


Choose for your Dawn House bed to be auto adjusted when snoring is detected.

- 5 times: Bed adjustment will be triggered when the snoring frequency reaches 5 times
- 10 times: Triggered when the snoring frequency reaches 10 times
- 15 times: Triggered when the snoring frequency reaches 15 times

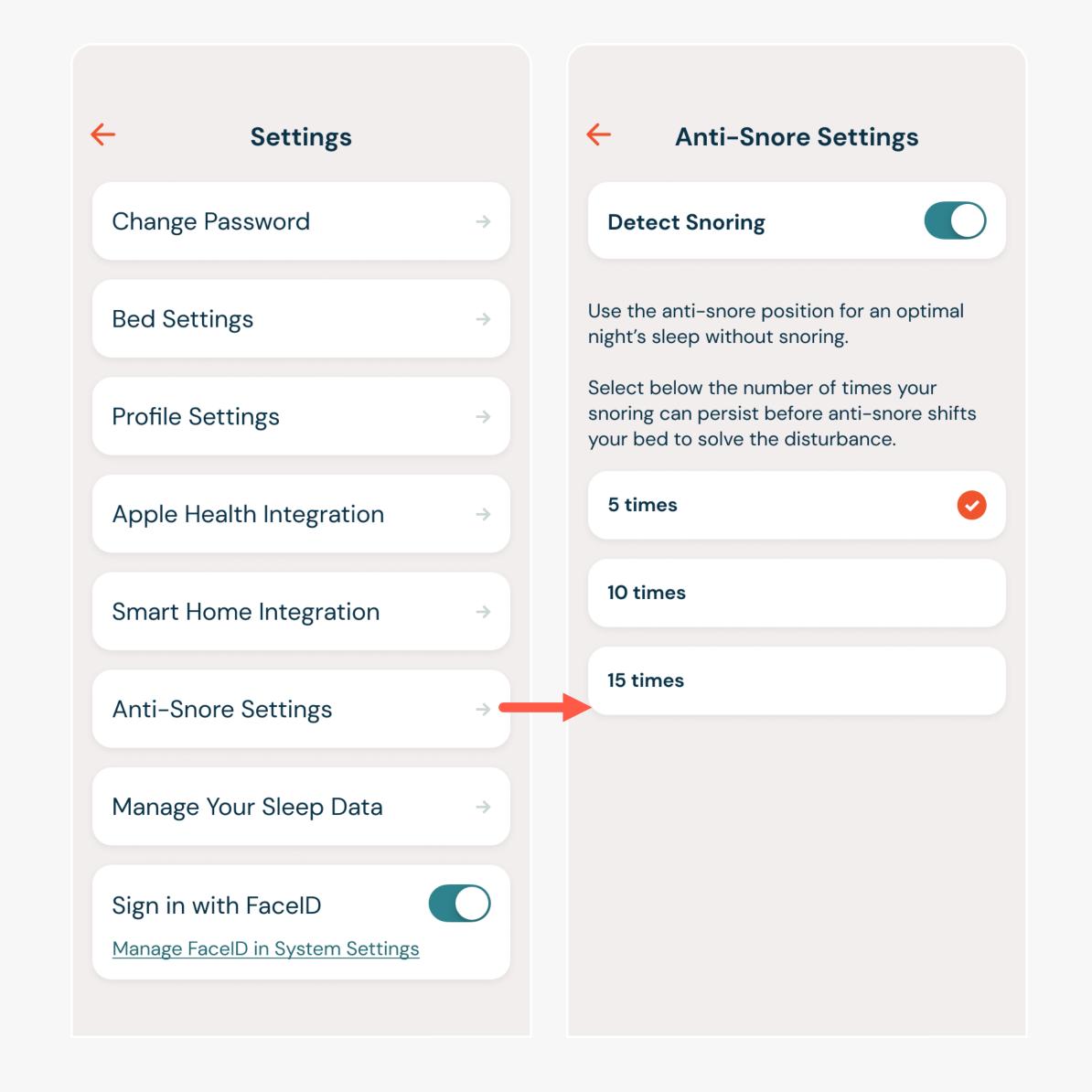


DELETE SLEEP DATA

To delete sleep data, select the button for "Delete Sleep Data". Next, select the orange icon for all data you'd like to delete. After, select the "Delete" button. You will see a confirmation modal to confirm your action.

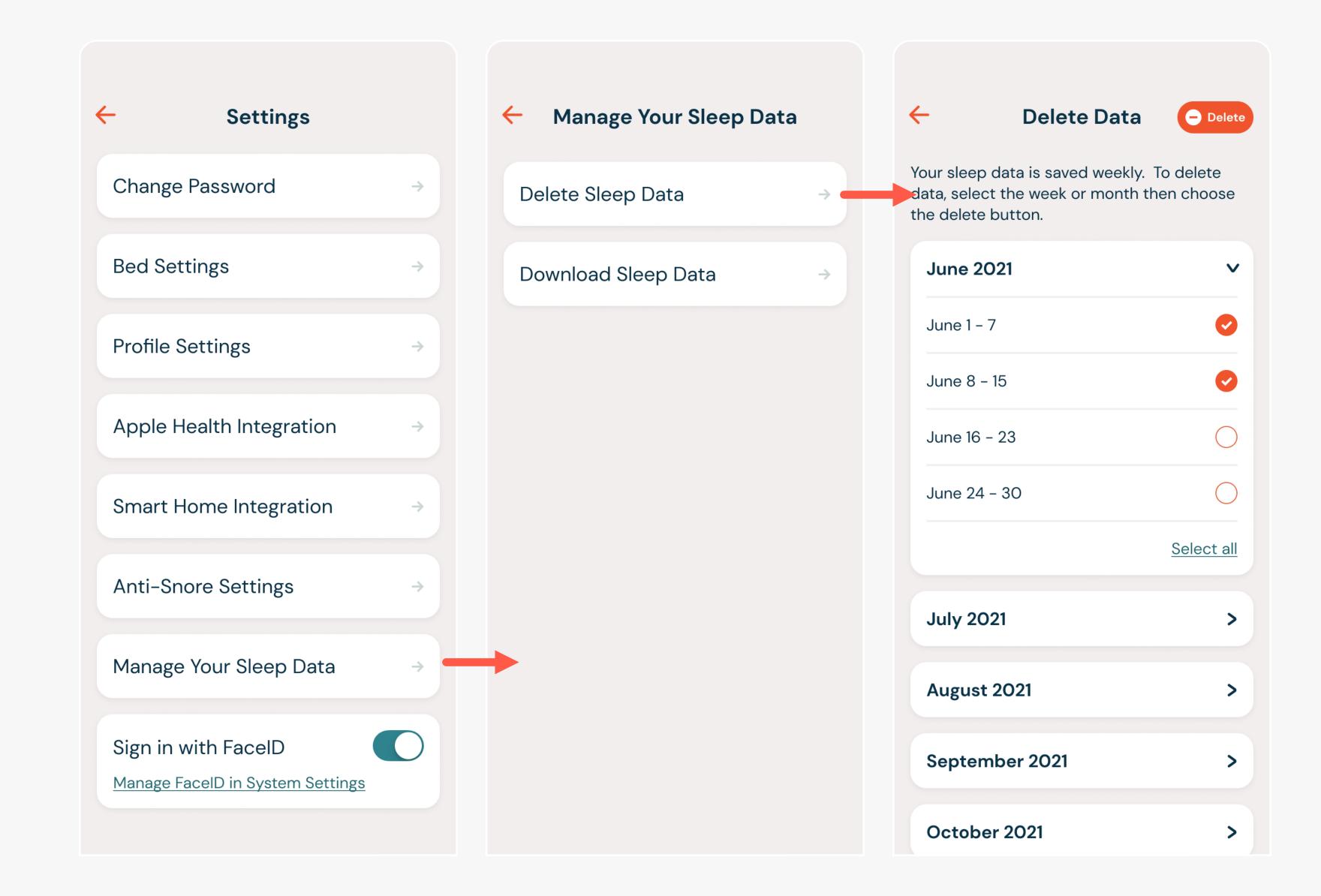

DOWNLOAD YOUR SLEEP DATA


To receive a copy of your sleep data, select the button for "Download Sleep Data". A copy of your sleep data will be sent to your email.

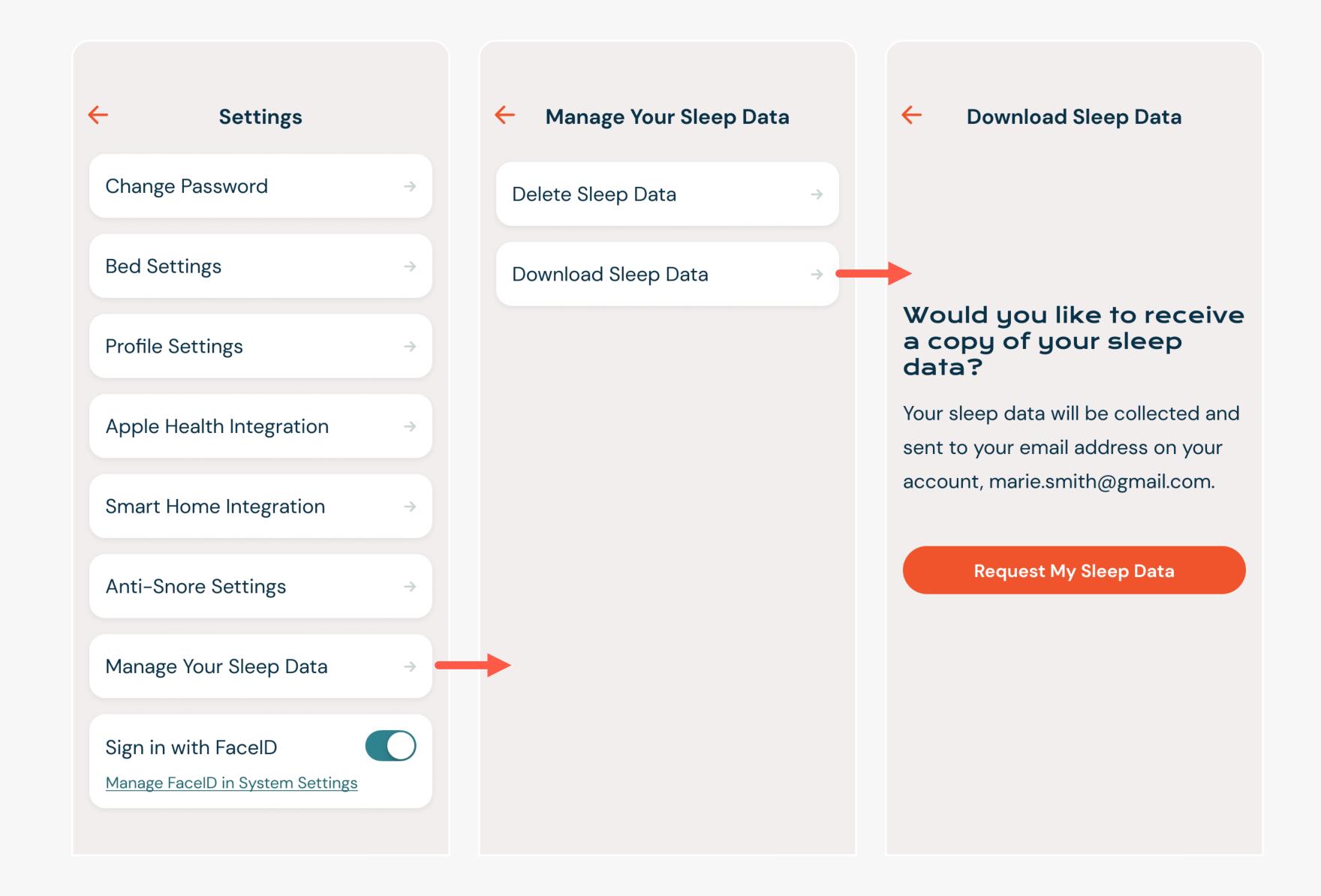



FACEID SETTINGS

You can log into the Dawn House app by using FaceID. This is authorized from the main Log In screen, but you can also turn FaceID on or off from the main App Settings screen.

If the toggle is in blue, FaceID is enabled. If the toggle is in grey, tap to enable FaceID. You will be prompted to enable the authorization and if needed, you will be linked to your Phone settings to confirm.

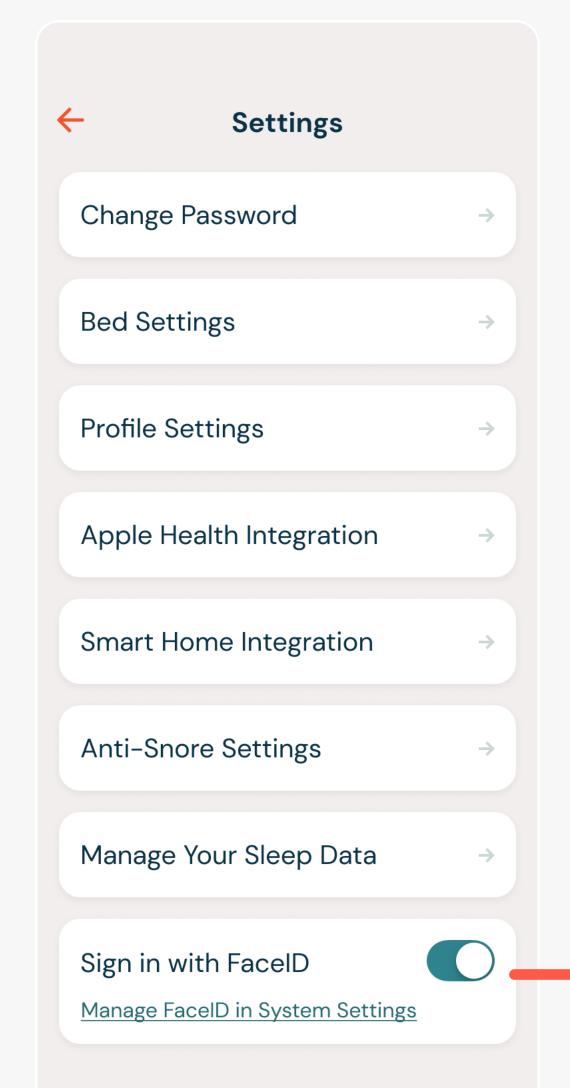

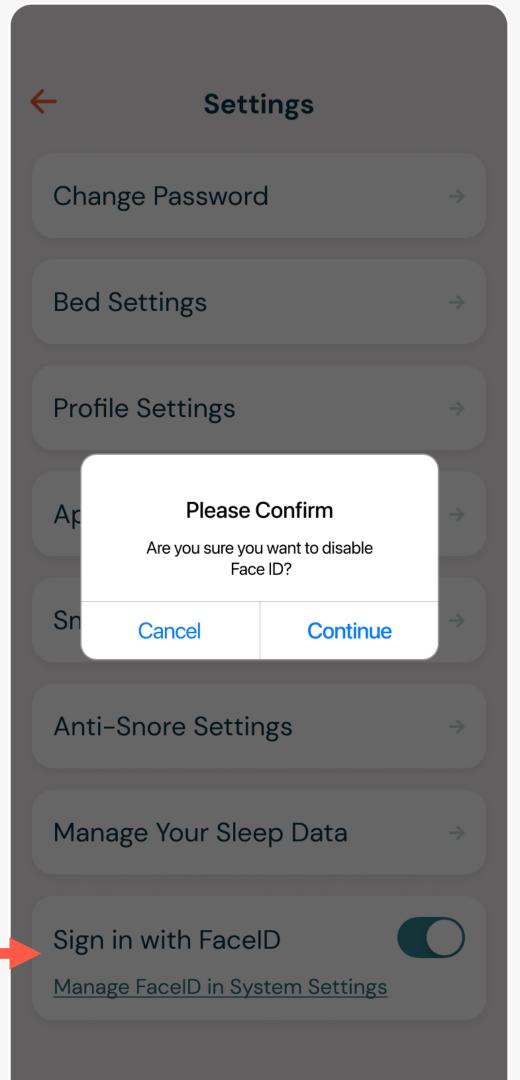

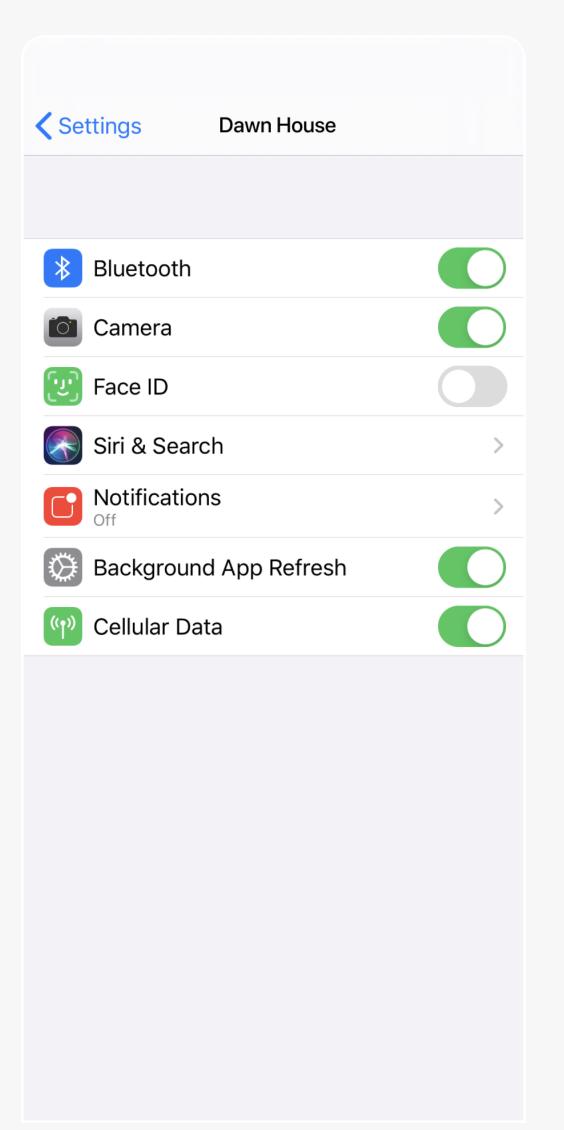

LOG OUT OF DAWN HOUSE

Click "Log Out" to log out of your Dawn House account.

You will need to re-enter your credentials or login in via FacelD to log back into the app.

