

INVACARE  
PLATINUM<sup>®</sup> MOBILE  
OXYGEN CONCENTRATOR  
**WITH CONNECTIVITY**



THE FUTURE OF OXYGEN IS HERE



Breathe Smart.  
Breathe Easy.



# Take charge of your day and get back to doing the things you love to do with the Invacare Platinum Mobile Oxygen Concentrator with Connectivity

- » Modern, discreet portable oxygen supply in a device made to weather daily use.
- » Easy-to-use app provides support at your fingertips and helps you go about your day.



The image features the Invacare Platinum Mobile Oxygen Concentrator, a black and white cylindrical device with a carrying strap. A smartphone is positioned in front of it, displaying the 'Piccolo O2' app interface. The app shows 'P2 Setting' with a bar graph, battery status, and a 'Time remaining' of 6:59. A cloud-shaped graphic above the device contains the text: INVACARE PLATINUM MOBILE OXYGEN CONCENTRATOR WITH CONNECTIVITY.

INVACARE  
**PLATINUM MOBILE**  
OXYGEN CONCENTRATOR  
WITH CONNECTIVITY



Three circular inset photos illustrate the device's use: a person hiking, a man laughing while wearing the device, and a couple smiling together. A red banner at the bottom right contains the text: STAY IN CONTROL.

GO YOUR OWN WAY

BE CONFIDENT

STAY IN CONTROL

# GO YOUR OWN WAY

Don't let your oxygen supply hold you back from doing the things you love.

**Choose** the Invacare Platinum Mobile Oxygen Concentrator with Connectivity



## LIGHTWEIGHT AND DURABLE

Compact, lightweight and rugged design let you go about your day in a variety of real life conditions

## LONG BATTERY TIME

The comfort of up to 10 hours of oxygen so you can keep going

## STYLISH, THREE-WAY CONVERTIBLE BAG

Convertible to backpack, messenger or purse style, you can match your bag to your occasion

\* Meets IP22 water resistance standard for exposure up to 10 minutes. \*\*10 hours with two batteries but no need to carry both batteries at once. Refer to Owner's Manual for product specifications and product use.

Compact, convenient and durable so you're as mobile as you want to be



# BE CONFIDENT

Enjoy fun times with friends or family with the comfort of your portable oxygen concentrator close to you, even if it's tucked away, almost invisible.

## IT'S EASY with Invacare's Platinum Mobile Oxygen Concentrator with Connectivity




### QUIET AND DISCREET

Let your portable oxygen concentrator blend into the background thanks to its quiet performance and **Piccolo O<sub>2</sub> App** (available at no charge). Your handy **Piccolo O<sub>2</sub> App** shows you actual time left in hours and minutes and conveniently replicates your portable oxygen concentrator's display on your phone screen, even when your portable oxygen concentrator is out of sight.



*App is optional but recommended.*

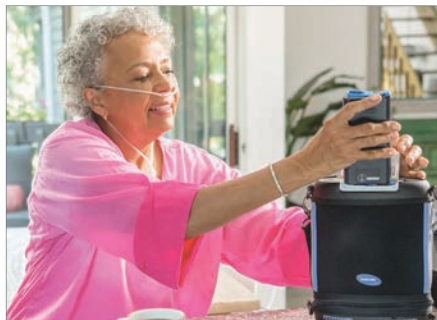

### NO OXYGEN INTERRUPTION

The Platinum Mobile Oxygen Concentrator's **"Hot swap"** capability lets you change or add batteries from the top without shutting down oxygen supply or removing the unit from the bag.



### EASY TO USE

**Large buttons** and a **simple, intuitive user interface** help you operate your Platinum Mobile Oxygen Concentrator with utmost ease.

No worries, just enjoy doing what you like to do


# STAY IN CONTROL

You're in charge of your day. Know actual battery time left and have easy access to quick guides for help, and support if you need it.

**IT'S POSSIBLE** with Invacare's Platinum Mobile Oxygen Concentrator with Connectivity



INFORMATION AND SUPPORT AT YOUR FINGERTIPS IN THE PICCOLO O<sub>2</sub> APP,  
NOW AVAILABLE FOR THE PLATINUM MOBILE OXYGEN CONCENTRATOR.

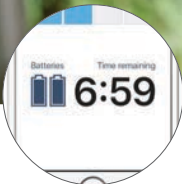

ACTUAL TIME LEFT  
IN HOURS/MINUTES



VIDEO  
RESOURCES



QUICK ACCESS  
TO USER MANUAL



HEALTH & LIFESTYLE  
RESOURCES



EASY ACCESS TO  
PROVIDER SUPPORT

You're empowered with knowledge and access to guides





# SUPPORT TO KEEP YOU GOING

Thanks to the Platinum Mobile Oxygen Concentrator with Connectivity

## OTHER BENEFITS OF YOUR PICCOLO O<sub>2</sub> APP

Your participating provider can assist in troubleshooting issues over the phone, potentially eliminating the need to wait for a technician to come to your home or for you to travel to your provider for assistance. This could mean less downtime for you and added confidence to go about your day.

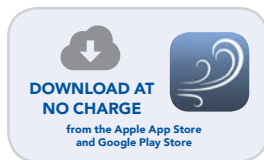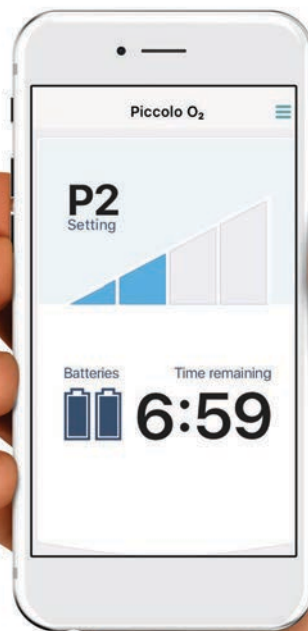

with Connectivity is available for you too.

## WELCOME TO THE FUTURE OF OXYGEN!

## PRODUCT SPECIFICATIONS

MODEL #: POC1-100B

Model #	Description
POC1-100B	Concentrator, battery, carrying case, AC adapter and DC adapter
POC1-100BA	Concentrator, carrying case, AC adapter and DC adapter, dual batteries
POC1-110	Supplemental battery - US
POC1-110-C	Supplemental battery - CA
POC1-115	Desktop battery charger- US
POC1-115-C	Desktop battery charger- CA
POC1-130	AC adapter
POC1-140	DC adapter 12V
POC1-150	Carrying case
POC1-CONNECT	Invacare USB Dongle, Platinum Mobile



App is Optional but recommended.



### Dimensions

9.45" H  
7.5" W  
3.88" D



### Oxygen Flow

1-4 Pulse Settings



### Maximum O2 Capacity

880 ml/min (@ setting 4)



### Limited Warranty

3 Year Concentrator  
1 Year Sieve Beds & Battery



### Battery Duration

One Battery: 5 hrs.  
Two Batteries: 10 hrs.  
(@ pulse setting 1)



### Operating Altitude

Up To 10,000 ft. above sea level



### FAA Compliant

Yes



### Weight

4.98 lb.\*\*\*  
15.9 oz. with Battery



### Oxygen Purity

87% - 95.6%



### DC Cable Input

12 V  
13.5 V Nominal



### AC Adapter Input

100-240 VAC  
50-60 Hz  
1.5 A - 0.6 A



### Recharge Time\*\*

One Battery: 2 hr 18 min  
Two Batteries: 4 hr 36 min



### Sound Pressure Level

≤40 dB (A) @ Setting 2 †

\* Meets IP22 water resistance standard for exposure up to 10 minutes

\*\* Recharge time increases if unit is running while charging.

\*\*\* Weight with single battery without carrying case

\*\*\*\*All references to HCPCS codes: Providers are responsible for determining the appropriate billing codes when submitting claims to the Medicare program, and should consult an attorney or other advisor to discuss specific situations in further detail.

Please refer to Owner's Manual for product specifications and product use.

† With carrying case

Concentrator or visit us online.



Breathe Smart.  
Breathe Easy.

[www.Invacare.com/BreatheSmartBreatheEasy](http://www.Invacare.com/BreatheSmartBreatheEasy) for more information.



## THE FUTURE OF OXYGEN IS HERE



Breathe Smart.  
Breathe Easy.

©2017 Invacare Corporation. All rights reserved. Trademarks are identified by the symbols <sup>TM</sup> and <sup>®</sup>. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted. Google Play and Android are trademarks of Google, Inc. Windows is a registered trademark of Microsoft Inc. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. Apple, iOS, and App Store are registered trademarks of Apple, Inc. Specifications are subject to change without notification. Form No. 17-270 170875